
THUASNE

Cervi-med

- D: HWS Stabilisierungsorthese mit Kehlkopfaussparung
GB/USA: Cervical stabilisation orthosis with a recess in the area of the larynx
F: Orthèse cervicale avec cavité dans la zone du larynx
I: Ortesi di sostegno cervicale con apertura tracheale
E: Órtesis estabilizadora de la columna cervical con escote en la región laringea
P: Órtese estabilizadora da coluna cervical com abertura para a laringe
NL: Stabiliseringsorthese voor de HWZ een uitsparing ter hoogte van de adamsappel
DK: Stabiliseringsortose til halshvirvelsøjlen med udsparring til strubehovedet
S: HWS stabiliseringsortos med stödskraftsförstärkning
FIN: Kaularangan stabilointitukiside kurkunpääsyvennyksellä
GR: Κηδεμόνας σταθεροποίησης HWS με εσοχή για τον λάρυγγα
CZ: Stabilizační ortéza krční páteře s vybráním v oblasti hrtanu
PL: KS orteza stabilizująca z otworem na wysokości tchawicy
EST: Kõriavaga HWS stabiliseerimisortoos
LV: Cervikāla stabilizējoša ortoze ar izgriezumu balsenei
LT: Stabilizuojantis kaklinės stuburo dalies įtvartas su išėma ties gerklomis
SK: Stabilizačná ortéza krčnej chrbtice s výrezom na hrtan
SLO: HWS stabilizacijska ortoza z odprtino za grlo
H: Nyaki gerincszakasz stabilizáló ortézis gégefőnyílással
RO: Orteză de stabilizare a coloanei vertebrale cervicale cu decupaj pentru laringe
BG: Стабилизираща ортеза за шийния отдел на гръбначния стълб с вдлъбнатост за ларинкса
HR: HWS stabilizacijska ortoza s udubljenjem za larinks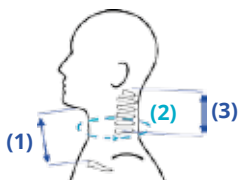

(1):	7,5 cm 3 inch	7,5 cm 3 inch	9,0 cm 3 ½ inch	10,5 cm 4 ½ inch
(2):	9,0 cm 3 ½ inch	10 cm 4 inch	12,0 cm 4 ¾ inch	14,5 cm 5 ½ inch

XS	25 – 32 cm 9 4/5 – 12 ½ inch	x	-	-	-
S	32 – 37 cm 12 ½ – 14 ½ inch	-	x	x	x
M	36 – 41 cm 14 1/5 – 16 1/5 inch	-	x	x	x
L	40 – 45 cm 15 ¾ – 17 ¾ inch	-	x	x	x
XL	44 – 49 cm 17 1/3 – 19 1/3 inch	-	x	x	x

(1)

(2)

Cervi-med (49200)

D

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GB

USA

Cervical stabilisation orthosis with a recess in the area of the larynx

Purpose

Cervi-med is a cervical orthosis that surrounds the neck but is open at the larynx. It is used to stabilise the cervical spine. The orthosis may only be used for cervical spine treatment.

Mode of action / Description

Cervi-med is a stabilising orthosis for stabilisation of the cervical spine and to relieve the strain on the musculature. Shaping in the regions of the chin, shoulder and ears, as well as a recess in the area of the larynx ensure optimum fit and high levels of wearing comfort. The excellent wearing comfort is added to by a skin-friendly cotton covering.

Indications

Whiplash; Cervical arthrosis; Sprain; Postoperatively; Torticollis; Cervical syndrome

Contraindications

Skin lesions or diseases in the affected area.

Side effects

None reported when applied and worn as intended.

Putting on the orthosis

1. Open the velcro fastening.
2. Place the orthosis around the neck so that the arrows **(1)** and the „T“ **(2)** on the fastening are pointing upwards.
3. Close the velcro fastening at the back of the neck. The stabilising orthosis should lie comfortably around the neck and be neither too tight nor too loose.

Precautions

Follow the advice of the professional who prescribed or supplied the product. Consult your professional if discomfort occurs. Store at room temperature, preferably in its original packaging. For hygiene and performance reasons, do not re-use the product for another patient. Any serious incident that occurs in connection with the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient are/is domiciled.

Care

Close the velcro fastening before washing. After washing, carefully press out the water and allow to air-dry.

Material composition

59 % polyurethane, 28 % cotton, 7 % polyethylene, 6 % polyamide

Please keep this instruction leaflet

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Indications

Coup de fouet cervical; Arthrose cervicale; Entorse; Postopératoire; Torticolis; Syndrome cervical

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- IL** THUASNE CERVITEX Ltd. · The Bell House, 20 Hataas St. · Kfar Saba, 44425
- S** THUASNE SCANDINAVIA AB · Box 95 · 131 07 Nacka
THUASNE DEUTSCHLAND GmbH