THUASNE

# Cervi-moll

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GB

USA

Cervical stabilisation orthosis made of foam

### Purpose

Cervi-moll is a cervical orthosis that surrounds the neck. It is used to stabilise the cervical spine. The orthosis may only be used for cervical spine treatment.

### Mode of action / Description

Cervi-moll is a stabilising orthosis made of foam providing medium support. It stabilises the cervical spine, relieves the strain on the musculature and supports the healing process. The cervical collar is anatomically shaped, especially in the regions of the chin and shoulders. Different sizes as well as an individually adjustable Velcro closure ensure optimum fit. The high level of wearing comfort is added to by a skin-friendly cotton covering.

### Indications

Whiplash; Cervical arthrosis; Sprain; Postoperatively; Torticollis; Cervical syndrome

### Contraindications

Skin lesions or diseases in the affected area.

### Side effects

None reported when applied and worn as intended.

### Putting on the orthosis

1. Place the orthosis around the neck so that the arrow **(1)** on the fastening is pointing upwards.
2. Close the velcro fastening at the back of the neck. The stabilising orthosis should lie comfortably around the neck and be neither too tight nor too loose.
3. You can wear Cervi-moll during the day and at night.

### Precautions

Follow the advice of the professional who prescribed or supplied the product. Consult your professional if discomfort occurs. Store at room temperature, preferably in its original packaging. For hygiene and performance reasons, do not re-use the product for another patient. Any serious incident that occurs in connection with the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient are/is domiciled.

### Care

Close the velcro fastening before washing. After washing, carefully press out the water and allow to air-dry.

### Material composition

73 % polyurethane, 21 % cotton, 6 % polyamide

Please keep this instruction leaflet

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### Indications

Coup de fouet cervical; Arthrose cervicale; Entorse; Postopératoire; Torticolis; Syndrome cervical

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- IL** THUASNE CERVITEX Ltd. · The Bell House, 20 Hataas St. · Kfar Saba, 44425
- S** THUASNE SCANDINAVIA AB · Box 95 · 131 07 Nacka
## THUASNE DEUTSCHLAND GmbH