THUASNE

# PDC

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Cervical spine immobilising orthosis with mobilising function

### **Purpose**

PDC is a cervical orthosis that surrounds the neck and incorporates a mobilisation function. It is used to immobilise the cervical spine. The orthosis may only be used for cervical spine treatment.

### **The concept**

The physiotherapeutic, dynamic cervical collar provides either bilateral or single sided immobilisation of the cervical spine. The main focus of product development was the concept of earliest possible functional treatment. The PDC should, as much as possible, prevent the painful disruption of mobility and promote free movement. Special consideration was given to the free front access to the neck, so that the PDC can be worn by patients who have an enlarged thyroid gland, or have undergone tracheostomy. The open-frame construction and the highly porous, air permeable foam padding achieve an excellent degree of patient acceptance and compliance.

### **Indications**

Changes due to wear and tear (degeneration), sprains and strains (distortions), postoperative.

### **Contraindications**

Skin lesions or diseases in the affected area.

### **Side effects**

None reported when applied and worn as intended.

### **Putting on the orthosis**

1. Place the orthosis around the neck so that the arrow on the fastening is pointing upwards.
2. Close the velcro fastening at the back of the neck. The stabilising orthosis should lie comfortably around the neck and be neither too tight nor too loose.

### **Exercises with the PDC**

It is the aim of these exercises to improve muscle tone, which has a positive effect upon the stabilisation of the cervical spine.

### **After consulting your doctor the following exercises are recommended:**

#### **(1) Lateral flexion**

Lower the head sideways left or right against the resistance of your own hand. Hold for approx. 20 seconds, then relax and repeat.

#### **(2) Rotation of the head**

When doing this exercise try and turn the head left or right against the resistance of either hand holding the chin. Here, too, you should keep up the pressure for 20 seconds, before pausing and repeating on the other side.

### **(3) Combined rotation/Eye movement exercise**

During this exercise the head is again turned left or right against the resistance of the chin held in the hand. In addition, however, the patient's eyes follow the direction of the attempted head movement. Press against the resistance for approx. 20 seconds, then relax and repeat.

### **(4) Lowering the head**

When bending the head forwards, press against the resistance of the collar for approx. 20 seconds.

### **(5-6) Single-sided freedom of movement**

The modular construction of the PDC enables the wearer to conduct single-sided mobilisation exercises. If, for example, movement to the right is blocked by the cervical spine, neck exercises are still possible on the left hand side after the removal of the modular insert. Thus, the affected side is immobilised whilst muscular dystrophy is avoided on the healthy side.

### **(7) Transformation for single-sided use**

- Roll back the central sleeve to one side
- Using a screwdriver, unclip the 4 retaining slots in the oval shaped module from the collar frame
- Replace the cover

### **Precautions**

Follow the advice of the professional who prescribed or supplied the product. Consult your professional if discomfort occurs. Store at room temperature, preferably in its original packaging. For hygiene and performance reasons, do not re-use the product for another patient. Any serious incident that occurs in connection with the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient are/is domiciled.

### **Care instructions**

Close the velcro fastening before washing. After washing, carefully press out the water and allow to air-dry.

### **Material composition**

47 % polyurethane, 19 % cotton, 19 % polycarbonate, 12 % polyethylene, 3 % polyamide

Please keep this instruction leaflet

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- IL** THUASNE CERVITEX Ltd. · The Bell House, 20 Hataas St. · Kfar Saba, 44425
- S** THUASNE SCANDINAVIA AB · Box 95 · 131 07 Nacka
## THUASNE DEUTSCHLAND GmbH