**THUASNE**

# Schulter-Fix Basic (SFB) Schulter-Fix Forte (SFF)

D: Schultergelenkorthese  
GB/USA: Shoulder fixation orthosis  
F: Orthèse de fixation de l'épaule  
I: Ortesi per l'immobilizzazione della spalla  
E: Órtesis para la fijación del hombro  
P: Órtese para a articulação do ombro  
NL: Schouderfixatieorthese  
DK: Skulderortose  
S: Axelledsortos  
FIN: Olkaniveltukiside  
GR: Νάρθηκας άρθρωσης ώμου  
CZ: Ortéza k fixaci ramenního kloubu  
PL: Orteza stawu ramiennego  
EST: Õlaliigese ortoos  
LV: Pleca locītavas ortoze  
LT: Peties sąnario įtvarymas  
SK: Ortéza ramenného kĺbu  
SLO: Orteze za ramenski sklep  
H: Vállfűzeti ortézis  
RO: Orteză pentru articulația umărului  
BG: Ортеза за раменна става  
HR: Ortoza za rameni zglob

	<b>Größe<sup>1</sup></b>	<b>Taillenumfang<sup>2</sup></b>
<b>48040</b>	100	< 95 cm < 37 ½ inch
	130	85 - 125 cm 33 ½ - 49 ½ inch
<b>48045</b>	130	85 - 125 cm 33 ½ - 49 ½ inch
	160	115 - 155 cm 45 ½ - 61 inch

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





<sup>1</sup>Size · Taille · Taglia · Talla · Tamanho · Maat · Størrelse · Storlek · Koko · Μέγεθος · Velikost · Rozmiar · Suurus · izmērs · Dydis · Velkosť · Velikost · Méret · Mārieme · Размер · Veličina

<sup>2</sup>Waist · Tour de taille · Circonferenza vita · Contorno de la cintura · Perímetro da cintura · Taille-omvang · Taljeomkreds · Midjemått · Vyötärön ympärys · Περίμετρος μέσης · Obvod pasu · Obwód talii · Talje ümbermõõt · vidukļa apkārtmērs · Liemens apimtis · Obvod pásu · Obseg pasu · Haskörfogat · circumferința talie · Обиколка на талията · Opseg struka

<sup>3</sup>Symmetric · Modèle unique, droite/gauche · Uguale per entrambi i lati · Simétrico · Lados iguais · Symmetrisch · Samme i begge sider · Symmetrisk · Sama molemmilla puolilla · Ίδιο και για τις δύο πλευρές · Universální pro levou i pravou nohu · Obie strony jednakowe · Võrdsete külgedega · Abas puses vienādas · Dvipusis · Obojstranný model, pravý/ľavý · Obe strani enaki · Kétoldalás, jobbos-balos modell · Laterale exacte · симетричен · Isto s obje strane

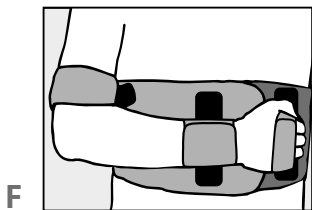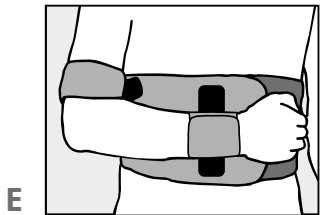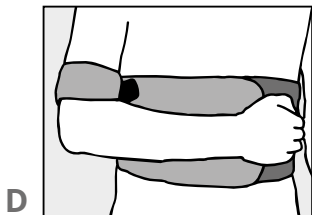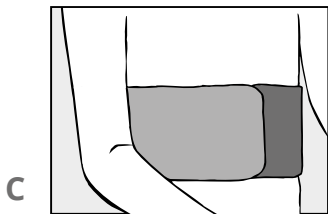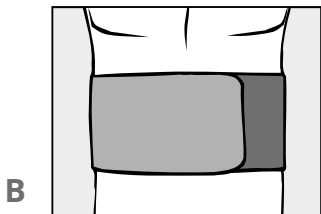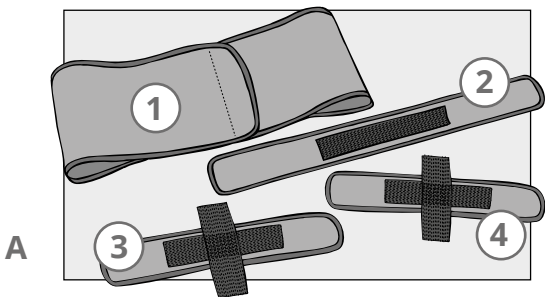## Schultergelenkorthese

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Shoulder fixation orthosis

### **Purpose**

Schulter-Fix Basic and Schulter-Fix Forte are orthoses for immobilising shoulder joints in fixed positions. The orthoses may only be used for shoulder treatment.

### **Mode of action/Description**

The shoulder fixation orthosis holds and fixes the arm in place at the shoulder joint in the position required medically. The orthosis can be individually adapted to the fitting requirements thanks to the numerous options for adjustment. All functions and fastenings are equipped with comfortable velcro systems that are easy to use.

### **Indications**

Shoulder injuries and diseases, (e.g. sprains, distortions), dislocations, post-operative.

### **Contraindications**

Larger skin lesions or diseases in the affected area, extended hematoma and severe swelling.

### **Side effects**

None reported when applied and worn as intended.

### **Putting on the orthosis**

- A. Take the orthosis out of the packaging and spread out all four parts separately.
- B. Place the torso belt (1) around the body below the chest and close the Velcro fastening at the front.
- C. Select the height of the torso belt so that afterwards (as described under D) the upper arm can be fixed to the torso belt (1) with the upper arm band (2). Here and in the following procedure ensure that it remains free of wrinkles.
- D. Place the upper arm band (2) around the upper arm directly above the elbow joint and close it. The outer Velcro strip must be positioned on the inside of the arm facing the body. Fixate the shoulder by attaching the upper arm band (2) to the torso belt (1) in accordance with the physician's instructions.
- E. Place the wrist band (3) around the wrist, close it and fix it to the torso belt (1) by means of the Velcro strip. The elbow joint should be positioned at a right angle.
- F. Close the hand-holding band (4) around the fingers of the hand to be fixated and fasten it by means of the Velcro strip on the torso belt (1).

## Precautions

Follow the advice of the professional who prescribed or supplied the product. Consult your professional if discomfort occurs. Store at room temperature, preferably in its original packaging. For hygiene and performance reasons, do not re-use the product for another patient. Any serious incident that occurs in connection with the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient are/is domiciled.

## Care instructions

Use only mild detergents. Close the Velcro fasteners before washing. After washing, carefully press out the water (do not squeeze) and allow to air-dry.

## Composition

**48040:** 50 % PA (polyamide) , 30 % PU (polyurethane), 15 % CO (cotton), 5 % PES (polyester)

**48045:** 38 % PA (polyamide), 25 % PE (polyethylene), 22 % PU (polyurethane), 10 % CO (cotton), 5 % PES (polyester)

**Prod. code 48040** Schulter-Fix Basic (SFB):  
shoulder fixation orthosis.

Torso belt 100 or 130 cm.

**Prod. code 48045** Schulter-Fix Forte (SFF):  
with reinforced torso belt (1).

Torso belt 130 or 160 cm.

- (1) Torso belt
- (2) Upper arm band
- (3) Wrist band
- (4) Hand holding band

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- IL** THUASNE CERVITEX Ltd. · The Bell House, 20 Hataas St. · Kfar Saba, 44425
- S** THUASNE SCANDINAVIA AB · Box 95 · 131 07 Nacka
## THUASNE DEUTSCHLAND GmbH