THUASNE

SILIMA®

D: Gebrauchsanweisung Post-OP Belt  
GB/USA: Instructions for use Post-OP Belt  
F: Notice d'utilisation pour ceinture Post-OP  
I: Istruzioni per l'uso Post-OP Belt  
E: Instrucciones de utilización banda postoperatoria  
P: Instruções de uso cinto pós-operatório  
NL: Gebruiksaanwijzing Post-OP Belt  
DK: Brugsanvisning Post-OP-belt  
S: Bruksanvisning Post-op-band  
FIN: Käyttöohje Post-OP Belt  
N: Bruksanvisning Post-OP belte  
GR: Οδηγίες χρήσης Μετεγχειρητική ζώνη  
CZ: Návod k použití pooperační fixační pás  
PL: Sposób użycia pas pooperacyjny  
EST: Kasutusjuhend operatsioonijärgne vöö  
LV: Lietošanas instrukcija pēcoperācijas josta  
LT: Naudojimo instrukcija pooperacinis diržas  
SK: Návod na použitie pooperačný pás  
SLO: Navodila za uporabo pooperacijski pas  
H: Használati utasítás operáció utáni öv  
RO: Instrucțiuni de utilizare centură postoperatorie  
BG: Инструкция за употреба Постоперативен колан  
HR: Uputa za uporabu Post-OP Belt  
MK: Упатство за користење Постоперативен појас  
SRB: Uputstvo za korišćenje Post-operativni pojas  
TR: Kullanım Talimatları Post-OP Belt  
RUS: Инструкция по применению Бандаж к бюстгалтеру послеоперационный компрессионный Post-OP Belt  
ROK: 사용 설명서 수술 후 벨트(Post-OP Belt)  
J: 使用方法 手術後用ベルト  
IL: Post-OP Belt הוראות שימוש  
AR: دليل الاستخدام حزام Post-OP

## D

### Post-OP Belt (Art.-Nr. 57 231)

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## **GB/USA**

### **Post-OP Belt (Prod. Code T57231)**

Anatomically shaped breast support belt with Velcro-type fastener

#### **Purpose & mode of operation:**

The Post-OP Belt secures breast implants and prevents them from riding up. The belt immobilises and stabilises the breast after surgery. The highly elastic and breathable Combitec fabric conforms well to the outline of the breast. The belt is anatomically shaped and supports the compression effect. The skin-friendly material is not bulky and the belt is easy to put on and take off.

Recommended for use together with SILIMA® Compression Bra Cara (Prod. Code 57 230); can also be used with other functional bras.

#### **Indications:**

- Breast reconstruction after mastectomy
- Breast augmentation with silicone implants
- After all plastic breast surgeries

#### **Contraindications:**

- Impairment of circulation or lymphatic soft-tissue swelling
- Allergic and inflammatory skin alterations (e.g. swelling, redness) in the affected area

<b>Size</b>	<b>Chest circumference</b> (measure above the breast and under the armpits)
S	86 – 92 cm (= 34 - 36 inches)
M	92 – 98 cm (= 36 - 38.5 inches)
L	98 – 108 cm (= 38.5 - 42.5 inches)
XL	108 – 118 cm (= 42.5 - 46.5 inches)
XXL	118 – 128 cm (= 46,5 - 50 inches)

Belt height: 10,5 cm (= 4 inches)

N.B.: You should consult your doctor or nurse on sizing, application procedure, length of treatment or any other issues.

#### **How to put on:**

The Post-OP Belt is closed over the top of the breast with a Velcro-type fastener:

- Hold the belt so that the garment label points inwards and is legible.
- Then close the belt from right to left.

#### **Precautions:**

Follow the advice of the professional who prescribed or supplied the product. This medical device is intended for the treatment of a single patient. To guarantee its efficacy and for hygiene reasons it is not allowed to pass it on to other patients. Any serious incident that occurs in connection with the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient are/is domiciled.

#### **Washing instructions:**

30°, no fabric softeners (see also garment label)

Close Velcro-type fastener before washing!

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- IL** THUASNE CERVITEX Ltd. · The Bell House, 20 Hataas St. · Kfar Saba, 44425
- S** THUASNE SCANDINAVIA AB · Box 95 · 131 07 Nacka
- CHN** THUASNE (Shanghai) Medical Devices Co., Ltd. · 159 Tianzhou Lu · Shanghai, 200233
**THUASNE DEUTSCHLAND GmbH**