**THUASNE**

# Epi-med®

- D: Epicondylitis-Spange
- GB/USA: Epicondylitis clasp
- F: Bracelet anti-épicondylite
- I: Pressore per epicondilita
- E: Pulsera para epicondilitis
- P: Braçadeira para epicondilita
- NL: Epicondylitis brace
- DK: Epicondylitis-bandage
- S: Epikondylitarmbågsstöd
- FIN: Tenniskyynärpäätuki
- GR: Περιαγκώνιο επικονδυλίτιδος
- CZ: Epikondylární páska
- PL: Orteza epikondylopatii
- EST: Põndapealisepõletiku pannal
- LV: Epikondilīta ortoze
- LT: Įtvaras nuo epikondilito
- SK: Epikondylická páska
- SLO: Epikondilitis zaponka
- H: Epicondylitis ortézis
- RO: Orteză epicondilită
- BG: Епикондиларна лента
- HR: Steznik za teniski lakat

Unterarmumfang · Forearm  
circumference · Tour de l'avant-bras<sup>1</sup>

XS	22 - 23 cm	8½ - 9"
S	24 - 25 cm	9¼ - 9¾"
M	26 - 27 cm	10¼ - 10½"
L	28 - 30 cm	11 - 11¾"
XL	31 - 34 cm	12¼ - 13½"

<sup>1</sup>Circonférence avambraccio · Contorno del antebrazo · Perímetro do antebraço · Omvang onderarm · Underarmsomkreds · Underarmsomkrets · Kyynärvarren ympärys · Περίμετρος αντιβραχίου · Obvod předloktí · Obwód przedramienia · Küünarvarre ümbermõõt · apakšdelma apkārtmērs · Dilbio apimtis · Obvod predlaktia · Obseg podlahti · Alkarkörfogat · circumferința antebrăț · Обиколка на ръката под лакътя · Opseg podlaktice

**Epi-med® (48001)**

**D**

## **Epicondylitis-Spange**

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## **Epicondylitis clasp**

### **Purpose:**

Epi-med® is an Epicondylitis clasp. The orthosis may only be used for elbow treatment. This product requires qualified and individual adaptation to the patient by specialist staff.

### **How it works:**

The Epi-med® epicondylitis clasp compresses the forearm musculature below the elbow, thus bringing mechanical relief to the tendon attachments at the epicondyle (bone projection) and relieving the extensor and flexor musculature. This biomechanical adjustment of the areas under strain in the elbow leads to a noticeable improvement, frequently to complete relief from pain.

### **Indications:**

- Tendinopathy (inflammation of the tendon attachments) in the elbow
- Medial epicondylitis (golfer's elbow)
- Lateral epicondylitis (tennis elbow)

### **Contraindications:**

Large skin lesions or diseases, extended hematomas or severe swelling.

### **Side effects:**

No side effects are as yet known for correct use.

### **Precautions:**

Follow the advice of the professional who prescribed or supplied the product. Consult your professional if discomfort occurs. Store at room temperature, preferably in its original packaging. For hygiene and performance reasons, do not re-use the product for another patient. Any serious incident that occurs in connection with the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient are/is domiciled.

**Directions for applying:**

Position the opened clasp from the outside onto the outer forearm so that the larger of the two pads rests on the upper side of the forearm and is turned slightly towards the body. The Epi-med® clasp is then closed using the elastic strap. The strap is fastened to the hook and loop fastener firmly but not tightly. The Epi-med® clasp should not exert tangible pressure when the musculature is relaxed, compression should be felt only when the forearm musculature is tensed.

**Care:**

Handwash. Carefully squeeze the water out and dry the padding away from a heat source.

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- IL** THUASNE CERVITEX Ltd. · The Bell House, 20 Hataas St. · Kfar Saba, 44425
- S** THUASNE SCANDINAVIA AB · Box 95 · 131 07 Nacka
## THUASNE DEUTSCHLAND GmbH