
THUASNE

Epi PROMASTER

D: Ellenbogengelenkbandage mit verstärktem propriozeptiven Effekt
GB/USA: Anti-epicondylitis bandage with reinforced proprioceptive effect
F: Coudière anti-épicondylite à effet proprioceptif renforcé
I: Gomitiera con effetto proprioceettivo rinforzato
E: Codera con efecto proprioceptivo reforzado
P: Órtese para cotovelo com efeito proprioceptivo reforçado
NL: Verband voor het ellebooggewricht met versterkt proprioceptief effect
DK: Albuebind med forstærket proprioceptiv virkning
S: Armbågsledsförband med förstärkt proprioceptiv verkan
FIN: Kynärside vahvistetulla proprioseptiivisellä vaikutuksella
GR: Επιπαικωνίδα με ενισχυμένη ιδιοδεκτικότητα
CZ: Bandáž lokte se zesíleným proprioceptivním efektem
PL: Orteza stawu łokciowego ze wzmocnionym efektem proprioceptywnym
EST: Küünarliigese bandaaž tugevdatud propriotseptiivse efektiga
LV: Elkoņa locītavas bandāža ar uzlabotu proprioceptīvo efektu
LT: Proprioceptinio poveikio sutvirtintas alkūnės įtvaras
SK: Ortéza lakťového kĺbu so zosilneným proprioceptívnym účinkom
SLO: Opora za komolec z izboljšanim proprioceptivnim učinkom
H: Könyökízület-rögzítő megerősített proprioceptív hatással
RO: Bandaj pentru încheietura cotului cu efect proprioceptiv pronunțat
BG: Ортеза за лакътна става с подсилено проприоцептивно действие
HR: Bandaža za zglob lakta s pojačanim proprioceptivnim učinkom

| Größe ¹ | Unterarmumfang ² | |
|--------------------|-----------------------------|-------------|
| | cm | inch |
| XS | 21 - 23 | 8 ¼ - 9 |
| S | 23 - 25 | 9 - 9 ¾ |
| M | 25 - 27 | 9 ¾ - 10 ½ |
| L | 27 - 29 | 10 ½ - 11 ½ |
| XL | 29 - 31 | 11 ½ - 12 ¼ |
| XXL | 31 - 34 | 12 ¼ - 13 ½ |

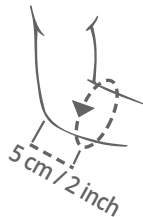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GB

USA

Anti-epicondylitis bandage with reinforced proprioceptive effect

Purpose:

Epi **PROMASTER** is a compression bandage with pads for treating elbow irritation. The bandage may only be used for elbow treatment.

Properties:

- Brace made of elastic knit
- 3D knit & pad opening at the condyles
- Epicondyle pads
- Latex-free

Indications:

Epicondylitis; Post-traumatic, postoperative and chronic inflammation; Sprains; Joint effusions and oedema linked to arthritis and joint disease

Contraindication:

Do not apply the product in direct contact with broken skin.

Side effects:

No side effects are as yet known for correct use.

Fitting instructions:

Make sure that the epicondyles are placed correctly in the special knit area. The brace should be worn directly against the skin.

Precautions:

Follow the advice of the professional who prescribed or supplied the product. Consult your professional if discomfort occurs. Store at room temperature, preferably in its original packaging. For hygiene and performance reasons, do not re-use the product for another patient. Any serious incident that occurs in connection with the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient are/is domiciled.

Washing instructions:

- Do not use detergents, fabric softeners or aggressive products (products containing chlorine)
- Squeeze out excess water
- Dry away from heat sources

Composition:

78 % PA (polyamide), 14 % EL (elastane), 5 % CO (cotton), 2 % PU (polyurethane), 1 % PES (polyester)

Explanation of the reinforced proprioceptive effect:

Orthoses in the **PROMASTER** range have been developed to improve the therapeutic effect - and, in particular, proprioception - on the joint. To achieve this, they activate special skin receptors on the joint and improve the latter's muscle support, thanks to innovative padding and an optimised compression effect.

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- IL** THUASNE CERVITEX Ltd. · The Bell House, 20 Hataas St. · Kfar Saba, 44425
- S** THUASNE SCANDINAVIA AB · Box 95 · 131 07 Nacka
THUASNE DEUTSCHLAND GmbH