THUASNE

Manu-GO

D: Handgelenkbandage mit propriozeptivem Effekt
GB/USA: Wrist immobilisation splint with proprioceptive effect
F: Attelle d'immobilisation du poignet à effet proprioceptif
I: Polsiera con effetto propriocettivo
E: Muñequera con efecto propioceptivo
P: Órtese para punho com efeito proprioceptivo
NL: Polsverband met proprioceptief effect
DK: Håndledsbind med proprioceptiv virkning
S: Handledsförband med proprioceptiv verkan
FIN: Ranneside proprioseptiivisellä vaikutuksella
GR: Πηχειοκάρπιο με ιδιοδεκτικότητα
CZ: Bandáž zápěstí se proprioceptivním efektem
PL: Orteza nadgarstkowo-ręczna ze efektem proprioceptywnym
EST: Randmeliigese bandaaž proprioceptiivse efektiga
LV: Plaukstas locītavas bandāža ar proprioceptīvo efektu
LT: Proprioceptinio poveikio riešo įtvaras
SK: Ortéza na zápästný kĺb so proprioceptívnym účinkom
SLO: Zapestni povoj z proprioceptivnim učinkom
H: Csuklórögzítő proprioceptív hatással
RO: Bandaj pentru încheietura mâinii cu efect proprioceptiv
BG: Ортеза за става на китка с proprioceptивно действие
HR: Bandaža za ručni zglobov s proprioceptivnim učinkom

Größe ¹	Handgelenkumfang ²	
	cm	inch
XS	14 - 15	5 ½ - 5 ⅞
S	15 - 16	5 ⅞ - 6 ¼
M	16 - 17	6 ¼ - 6 ¾
L	17 - 18	6 ¾ - 7
XL	18 - 19	7 - 7 ½
XXL	19 - 20	7 ½ - 7 ⅞

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Wrist immobilisation splint with proprioceptive effect**Purpose:**

Manu-GO is a compression bandage with pad for treating wrist irritation. The bandage may only be used for wrist treatment.

Properties:

- Brace made of elastic knit
- 3D knit & pad opening at the condyle
- Stabilisation stay
- Stabilisation strap
- Latex-free

Indications:

Tendinitis and tendon disorders of the wrist; Post-traumatic, postoperative and chronic inflammation; Sprains; Joint effusions and oedema linked to arthritis and joint disease

Contraindication:

Do not apply the product in direct contact with broken skin.

Side effects:

No side effects are as yet known for correct use.

Fitting instructions:

Shape the stabilisation stay if necessary. Make sure that the thumb is placed correctly in its opening. Close the stabilisation strap. The brace should be worn directly against the skin.

Precautions:

Follow the advice of the professional who prescribed or supplied the product. Consult your professional if discomfort occurs. Store at room temperature, preferably in its original packaging. For hygiene and performance reasons, do not re-use the product for another patient. Any serious incident that occurs in connection with the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient are/is domiciled.

Washing instructions:

- Do not use detergents, fabric softeners or aggressive products (products containing chlorine)
- Squeeze out excess water
- Dry away from heat sources

**Composition:**

86 % PA (polyamide), 14 % EL (elasthane)

Explanation of the proprioceptive effect:

Orthoses in the GO range have been developed to improve the therapeutic effect - and, in particular, proprioception - on the joint. To achieve this, they activate special skin receptors on the joint and improve the latter's muscle support, thanks to innovative padding and an optimised compression effect.

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- IL** THUASNE CERVITEX Ltd. · The Bell House, 20 Hataas St. · Kfar Saba, 44425
- S** THUASNE SCANDINAVIA AB · Box 95 · 131 07 Nacka
THUASNE DEUTSCHLAND GmbH