THUASNE

Lomba-GO

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Lumbar support belt with proprioceptive effect

Purpose:

Lomba-GO is a lumbar spine stabilising orthosis with back pad for treating lumbar pain. The orthosis may only be used for lumbar spine treatment.

Properties:

- Brace made of elastic knit
- 2 stabilisation stays each at front and back
- Possibility to fix a lumbar pad
- Grip pouch for easier fastening
- Latex-free

Indications:

Acute, sub-acute or chronic lumbar pain with or without nerve pain; Post-operative lumbar and pelvic support; Individuals with a connective tissue disease (e.g.: Ehlers-Danlos syndrome); Fibromyalgia

Contraindication:

Do not apply the product in direct contact with broken skin.

Side effects:

No side effects are as yet known for correct use.

Fitting instructions:

Put on and close the belt with the help of the grip pouch at the left side.

Precautions:

Follow the advice of the professional who prescribed or supplied the product. Consult your professional if discomfort occurs. Store at room temperature, preferably in its original packaging. For hygiene and performance reasons, do not re-use the product for another patient. Any serious incident that occurs in connection with the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient are/is domiciled.

Washing instructions:

- Do not use detergents, fabric softeners or aggressive products (products containing chlorine)
- Squeeze out excess water
- Dry away from heat sources

Composition:

86 % PA (polyamide), 14 % EL (elasthane)

Explanation of the proprioceptive effect:

Orthoses in the GO range have been developed to improve the therapeutic effect - and, in particular, proprioception - on the joint. To achieve this, they activate special skin receptors on the joint and improve the latter's muscle support, thanks to innovative padding and an optimised compression effect.

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- IL** THUASNE CERVITEX Ltd. · The Bell House, 20 Hataas St. · Kfar Saba, 44425
- S** THUASNE SCANDINAVIA AB · Box 95 · 131 07 Nacka
THUASNE DEUTSCHLAND GmbH