
THUASNE

Genu-GO

D: Patellabandage mit propriozeptivem Effekt
GB/USA: Patellar knee brace with proprioceptive effect
F: Genouillère à effet proprioceptif
I: Ginocchiera rotulea con effetto proprioceptivo
E: Rodillera rotuliana con efecto proprioceptivo
P: Joelheira rotuliana com efeito proprioceptivo
NL: Knieverband voor de knieschijf met proprioceptieve werking
DK: Knæbind med proprioceptiv virkning
S: Knäskålsförband med proprioceptiv verkan
FIN: Polviside proprioseptiivisellä vaikutuksella
GR: Συνδεσμική επιγονατίδα με ιδιοδεκτική δράση
CZ: Čéšková kolenní ortéza se proprioceptivním účinkem
PL: Orteza stawu kolanowego ze efektem proprioceptywnym
EST: Põlvekedrabandaaž proprioseptiivse efektiga
LV: Ceļa kaula bandāža ar proprioceptīvo efektu
LT: Kelio ginkelės įtvaras su proprioceptiniu poveikiu
SK: Kolenná ortéza so proprioceptívnym účinkom
SLO: Kolenski opornica za pogačico z proprioceptivnim učinkom
H: Patella védő térdrögztítő proprioceptív hatással
RO: Genunchieră rotuliană cu efect proprioceptiv
BG: Пателарна колянна ортеза с проприоцептивно действие
HR: Bandaža za patelu (čašicu koljena) s proprioceptivnim učinkom

Größe ¹	Oberschenkelumfang ²		Wadenumfang ³	
	cm	inch	cm	inch
XXS	35 – 38	13 ¾ - 15	25 – 28	9 ¾ - 11
XS	38 – 41	15 - 16	28 – 31	11 - 12 ¼
S	41 – 44	16 - 17 ¼	31 – 34	12 ¼ - 13 ½
M	44 – 47	17 ¼ - 18 ½	34 – 37	13 ¼ - 14 ⅔
M+	47 – 50	18 ½ - 19 ¾	34 – 37	13 ¼ - 14 ⅔
L	47 – 50	18 ½ - 19 ¾	37 – 40	14 ⅓ - 15 ¾
L+	50 – 53	19 ¾ - 21	37 – 40	14 ⅓ - 15 ¾
XL	50 – 53	19 ¾ - 21	40 – 43	15 ¾ - 17
XL+	53 – 56	21 - 22	40 – 43	15 ¾ - 17
XXL	53 – 56	21 - 22	43 – 46	17 - 18
XXL+	56 – 59	22 - 23 ¼	43 – 46	17 - 18

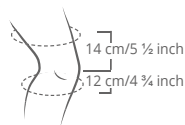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Patellar knee brace with proprioceptive effect**Purpose:**

Genu-GO is a compression bandage with patella pad for treating knee irritation. The bandage may only be used for knee treatment.

Properties:

- Brace made of elastic knit
- 3D knit at the patella
- Highly elastic knit for comfort of the popliteal fossa
- Patellar pad
- Integrated lateral reinforcements
- Latex-free

Indications:

Post-traumatic, postoperative and chronic irritation; Fibromyalgia and sprains of the lateral ligaments; Osteo- and rheumatoid arthritis; Mild joint instabilities; Patellar deficiencies

Contraindication:

Do not apply the product in direct contact with broken skin.

Side effects:

No side effects are as yet known for correct use.

Fitting instructions:

Make sure that the kneecap is properly centered in the central patellar ring. The brace should be worn directly against the skin.

Precautions:

Follow the advice of the professional who prescribed or supplied the product. Consult your professional if discomfort occurs. Store at room temperature, preferably in its original packaging. For hygiene and performance reasons, do not re-use the product for another patient. Any serious incident that occurs in connection with the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient are/is domiciled.

Washing instructions:

- Do not use detergents, fabric softeners or aggressive products (products containing chlorine)
- Squeeze out excess water
- Dry away from heat sources

**Composition:**

86 % PA (polyamide), 14 % EL (elasthane)

Explanation of the proprioceptive effect:

Orthoses in the GO range have been developed to improve the therapeutic effect - and, in particular, proprioception - on the joint. To achieve this, they activate special skin receptors on the joint and improve the latter's muscle support, thanks to innovative padding and an optimised compression effect.

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- IL** THUASNE CERVITEX Ltd. · The Bell House, 20 Hataas St. · Kfar Saba, 44425
- S** THUASNE SCANDINAVIA AB · Box 95 · 131 07 Nacka
THUASNE DEUTSCHLAND GmbH