THUASNE

Manu PROMASTER

D: Handgelenkbandage mit verstärktem propriozeptiven Effekt
GB/USA: Wrist immobilisation splint with reinforced proprioceptive effect
F: Attelle d'immobilisation du poignet à effet proprioceptif renforcé
I: Polsiera con effetto propriocettivo rinforzato
E: Muñequera con efecto propioceptivo reforzado
P: Órtese para punho com efeito propioceptivo reforçado
NL: Polsverband met versterkt proprioceptief effect
DK: Håndledsbind med forstærket proprioceptiv virkning
S: Handledsförband med förstärkt proprioceptiv verkan
FIN: Ranneside vahvistetulla proprioseptiivisellä vaikutuksella
GR: Πηχειοκάρπιο με ενισχυμένη ιδιοδεκτικότητα
CZ: Bandáž zápěstí se zesíleným proprioceptivním efektem
PL: Orteza nadgarstkowo-ręczna ze wzmocnionym efektem proprioceptywnym
EST: Randmeliigese bandaaž tugevdatud propriotseptiivse efektiga
LV: Plaukstas locītavas bandāža ar uzlabotu proprioceptīvo efektu
LT: Proprioceptinio poveikio sutvirtintas riešo įtvaras
SK: Ortéza na zápästný kĺb so zosilneným proprioceptívnym účinkom
SLO: Zapestni povoj z izboljšanim proprioceptivnim učinkom
H: Csuklórögzítő megerősített proprioceptív hatással
RO: Bandaj pentru încheietura mâinii cu efect propioceptiv pronunțat
BG: Ортеза за става на китка с подсилено проприоцептивно действие
HR: Bandaža za ručni zglobov s pojačanim proprioceptivnim učinkom

Größe ¹	Handgelenkumfang ²	
	cm	inch
XS	14 - 15	5 ½ - 5 ⅞
S	15 - 16	5 ⅞ - 6 ¼
M	16 - 17	6 ¼ - 6 ¾
L	17 - 18	6 ¾ - 7
XL	18 - 19	7 - 7 ½
XXL	19 - 20	7 ½ - 7 ⅞

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Wrist immobilisation splint with reinforced proprioceptive effect

Purpose:

Manu **PROMASTER** is a compression bandage with pad for treating wrist irritation. The bandage may only be used for wrist treatment.

Properties:

- Brace made of elastic knit
- 3D knit & pad opening at the condyle
- Stabilisation stay
- Stabilisation strap
- Latex-free

Indications:

Tendinitis and tendon disorders of the wrist; Post-traumatic, postoperative and chronic inflammation; Sprains; Joint effusions and oedema linked to arthritis and joint disease

Contraindication:

Do not apply the product in direct contact with broken skin.

Side effects:

No side effects are as yet known for correct use.

Fitting instructions:

Shape the stabilisation stay if necessary. Make sure that the thumb is placed correctly in its opening. Close the stabilisation strap. The brace should be worn directly against the skin.

Precautions:

Follow the advice of the professional who prescribed or supplied the product. Consult your professional if discomfort occurs. Store at room temperature, preferably in its original packaging. For hygiene and performance reasons, do not re-use the product for another patient. Any serious incident that occurs in connection with the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient are/is domiciled.

Washing instructions:

- Do not use detergents, fabric softeners or aggressive products (products containing chlorine)
- Squeeze out excess water
- Dry away from heat sources

Composition:

78 % PA (polyamide), 14 % EL (elastane), 5 % CO (cotton), 2 % PU (polyurethane), 1 % PES (polyester)

Explanation of the reinforced proprioceptive effect:

Orthoses in the **PROMASTER** range have been developed to improve the therapeutic effect - and, in particular, proprioception - on the joint. To achieve this, they activate special skin receptors on the joint and improve the latter's muscle support, thanks to innovative padding and an optimised compression effect.

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- IL** THUASNE CERVITEX Ltd. · The Bell House, 20 Hataas St. · Kfar Saba, 44425
- S** THUASNE SCANDINAVIA AB · Box 95 · 131 07 Nacka
THUASNE DEUTSCHLAND GmbH