
THUASNE

Reversa PROMASTER

D: Lumbalbandage mit verstärktem propriozeptiven Effekt
GB/USA: Lumbar support belt with reinforced proprioceptive effect
F: Ceinture de soutien lombaire à effet proprioceptif renforcé
I: Cintura di sostegno lombare con effetto proprioceptivo rinforzato
E: Faja de sujeción lumbosacra con efecto proprioceptivo reforzado
P: Cinta de sustentação lombar com efeito proprioceptivo reforçado
NL: Compacte steungordel voor de lende met versterkt proprioceptief effect
DK: Lændestøttebælte med forstærket proprioceptiv virkning
S: Ländryggsförband med förstärkt proprioceptiv verkan
FIN: Lanneside vahvistetulla proprioseptiivisellä vaikutuksella
GR: Συμπαιγής ζώνη οσφυϊκής υποστήριξης με ενισχυμένη ιδιοδεκτικότητα
CZ: Lumbální bandáž se zesíleným proprioceptivním efektem
PL: Orteza lędźwiowo-krzyżowa ze wzmocnionym efektem proprioceptywnym
EST: Lumbaalandaaž tugevdatud propriotseptiivse efektiga
LV: Lumbālā bandāža ar uzlabotu proprioceptīvo efektu
LT: Proprioceptinio poveikio sutvirtintas juosmens diržas
SK: Bedrová ortéza so zosilneným proprioceptívnym účinkom
SLO: Ledveni povoj z izboljšanim proprioceptivnim učinkom
H: Ágyéki fűző megerősített proprioceptív hatással
RO: Centură de susținere lombară cu efect proprioceptiv pronunțat
BG: Компактен лумбален колан с подсилено проприоцептивно действие
HR: Bandaža za lumbalni dio leđa s pojačanim proprioceptivnim učinkom

Leibumfang²

Größe ¹	cm	inch
XS	70 - 80	6 ² / ₃ - 7 ¹ / ₂
S	80 - 90	7 ¹ / ₂ - 8 ¹ / ₃
M	90 - 100	8 ¹ / ₃ - 9
L	100 - 110	9 - 9 ³ / ₄
XL	110 - 120	9 ³ / ₄ - 10 ¹ / ₂
XXL	120 - 130	10 ¹ / ₂ - 11 ¹ / ₂

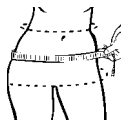¹Size · Taille · Taglia · Talla · Tamanho · Maat · Størrelse · Storlek · Koko · Μέγεθος · Velikost · Rozmiar · Suurus · izmērs · Dydis · Veľkosť · Velikost · Méret · Mărimē · Размер · Veličina

² Body circumference · Tour de l'abdomen · Circonferenza tronco · Contorno del vientre · Perímetro do ventre · Lijfomvang · Legemsomkreds · Livmått · Kehon ympärys · Περίμετρος κοιλιακής χώρας · Obvod břicha · Obwód w pasie · Keha ümbermõõt · vēdera apkārtmērs · Klubų apimtis · Obvod brucha · Obseg trupa · Derékkörfoga · circumferință trunchi · Обиколка на ханша · Opseg abdomena

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Lumbar support belt with reinforced proprioceptive effect

Purpose:

Reversa PROMASTER is a lumbar spine stabilising orthosis with back pad for treating lumbar pain. The orthosis may only be used for lumbar spine treatment.

Properties:

- Brace made of elastic knit
- 2 stabilisation stays each at sides and back
- Possibility to fix a lumbar pad
- Grip pouch for easier fastening
- Latex-free

Indications:

Acute, sub-acute or chronic lumbar pain with or without nerve pain; Post-operative lumbar and pelvic support; Individuals with a connective tissue disease (e.g.: Ehlers-Danlos syn-drome); Fibromyalgia

Contraindication:

Do not apply the product in direct contact with broken skin.

Side effects:

No side effects are as yet known for correct use.

Fitting instructions:

Put on and close the belt with the help of the grip pouch at the left side.

Precautions:

Follow the advice of the professional who prescribed or supplied the product. Consult your professional if discomfort occurs. Store at room temperature, preferably in its original packaging. For hygiene and performance reasons, do not re-use the product for another patient. Any serious incident that occurs in connection with the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient are/is domiciled.

Washing instructions:

- Do not use detergents, fabric softeners or aggressive products (products containing chlorine)
- Squeeze out excess water
- Dry away from heat sources

Composition:

78 % PA (polyamide), 14 % EL (elastane), 5 % CO (cotton), 2 % PU (polyurethane), 1 % PES (polyester)

Explanation of the reinforced proprioceptive effect:

Orthoses in the PROMASTER range have been developed to improve the therapeutic effect - and, in particular, proprioception - on the joint. To achieve this, they activate special skin receptors on the joint and improve the latter's muscle support, thanks to innovative padding and an optimised compression effect.

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- IL** THUASNE CERVITEX Ltd. · The Bell House, 20 Hataas St. · Kfar Saba, 44425
- S** THUASNE SCANDINAVIA AB · Box 95 · 131 07 Nacka
THUASNE DEUTSCHLAND GmbH