THUASNE

# Malleo PROMASTER

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GB

USA

## **Ankle brace with reinforced proprioceptive effect**

### **Purpose:**

Malleo **PROMASTER** and Malleo **PROMASTER Aktiv** are compression bandages with pads for treating ankle irritation. The bandages may only be used for ankle treatment.

### **Properties:**

- Brace made of elastic knit
- 3D knit at the malleoli
- Malleolar pads
- Latex-free

### **Indications:**

Post-traumatic, postoperative and chronic inflammation; Sprains; Joint effusions and oedema linked to arthritis and joint disease; Moderate joint instability

### **Contraindication:**

Do not apply the product in direct contact with broken skin.

### **Side effects:**

No side effects are as yet known for correct use.

### **Fitting instructions:**

Make sure that the two malleoli are positioned in the 3D knit zones specially designed for this purpose. The ankle brace can be worn against the skin or on top of a well fitting sock.

### **Precautions:**

Follow the advice of the professional who prescribed or supplied the product. Consult your professional if discomfort occurs. Store at room temperature, preferably in its original packaging. For hygiene and performance reasons, do not re-use the product for another patient. Any serious incident that occurs in connection with the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient are/is domiciled.

### **Washing instructions:**

- Do not use detergents, fabric softeners or aggressive products (products containing chlorine)
- Squeeze out excess water
- Dry away from heat sources

### **Composition:**

78 % PA (polyamide), 14 % EL (elastane), 5 % CO (cotton), 2 % PU (polyurethane), 1 % PES (polyester)

### **Explanation of the reinforced proprioceptive effect:**

Orthoses in the **PROMASTER** range have been developed to improve the therapeutic effect - and, in particular, proprioception - on the joint. To achieve this, they activate special skin receptors on the joint and improve the latter's muscle support, thanks to innovative padding and an optimised compression effect.

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- IL** THUASNE CERVITEX Ltd. · The Bell House, 20 Hataas St. · Kfar Saba, 44425
- S** THUASNE SCANDINAVIA AB · Box 95 · 131 07 Nacka
**THUASNE DEUTSCHLAND GmbH**