THUASNE

Pero-med AFO III/II

D: Fußheberorthese

GB/USA: Peroneal orthosis

F: Releveur de pied

I: Ortesi peroneale

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Peroneal orthosis

Purpose

Pero-med AFO III/II is a thermoplastically flexible foot orthosis for correction and/or pressure relief. The orthosis may only be used for foot treatment.

Indications

Mild to medium foot lifting weakness, mild peroneal pareses of the IIIrd and IInd degree without spasticity

Contraindications

Skin lesions or diseases in the affected area, especially with inflammatory symptoms. Sensation and blood circulation disturbances of the foot. Lymph flow disturbances, soft tissue swellings of unknown origin. Forefoot or heel ulcer.

Side effects

No side effects are as yet known for correct use.

Application instructions

- Open the calf Velcro strap.
- Insert the orthosis into the open shoe.
- Put the shoe on and position the lower leg in the orthosis
- Guide the calf strap round the calf under slight tension and close it.
- Close the shoe. The Pero-med AFO does only work in combination with a well-fitting shoe.

Precautions

Follow the advice of the professional who prescribed or supplied the product. Consult your professional if discomfort occurs. Store at room temperature, preferably in its original packaging. For hygiene and performance reasons, do not re-use the product for another patient. Any serious incident that occurs in connection with the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient are/is domiciled.

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- IL** THUASNE CERVITEX Ltd. · The Bell House, 20 Hataas St. · Kfar Saba, 44425
- S** THUASNE SCANDINAVIA AB · Box 95 · 131 07 Nacka
THUASNE DEUTSCHLAND GmbH