THUASNE

Sohli

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GB/USA: Foot support for children

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USA

Purpose:

The Sohli foot support is a flexible heel cup for children. The heel cup may only be used for foot treatment.

Mode of action:

Strong support at the Sustentaculum tali medially straightens the heel bone. Combined with an adverse positioning in the heel contact area, the Sohli foot supports compensate for the pes valgus. The heel reinforcement runs horizontally, the lateral space between foot and shoe is filled by an elastic support edge. The foot is firmly embedded and pain is eliminated.

Indication:

Pes valgus in children

Contraindications:

Do not apply the product in direct contact with broken skin.

Side effects:

No side effects are as yet known for correct use.

How to place the Sohlis:

- Take out first the original insole from the children's shoe first.
- Put the Sohli into the children's shoe and make sure it's even and fits well.

Precautions:

Follow the advice of the professional who prescribed or supplied the product. Consult your professional if discomfort occurs. Store at room temperature, preferably in its original packaging. For hygiene and performance

reasons, do not re-use the product for another patient. Any serious incident that occurs in connection with the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient are/is domiciled.

Care:

- Wash by hand
- Do not use detergents, fabric softeners or aggressive products (products containing chlorine)
- Squeeze out excess water
- Dry away from heat sources

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- IL** THUASNE CERVITEX Ltd. · The Bell House, 20 Hataas St. · Kfar Saba, 44425
- S** THUASNE SCANDINAVIA AB · Box 95 · 131 07 Nacka
THUASNE DEUTSCHLAND GmbH