THUASNE

Sleeq

D: Überbrückungsorthese
GB/USA: Bridging orthosis
F: Ceinture de soutien lombaire
I: Ortesi con sistema a ponte
E: Órtesis puente
P: Órtese de transição
NL: Overbruggingsorthese
DK: Overgangsortose
S: Överbrygningsortos
FIN: Selkäortoosi
GR: Νάρθηκας γεφύρωσης
CZ: Podpurná ortéza zad
PL: Orteza łądźwiowo-krzyżowa
EST: Tugiortoos
LV: Stabilizējoša ortoze
LT: Juosmens įtvaras
SK: Podporná chrbtová ortéza
SLO: Premostitvena ortoza
H: Áthidaló ortézis
RO: Orteză lombară
BG: Подпомагаща ортеза
HR: Premosna ortoza


1

SLEEQ

Klettverbindung (halbmondförmig)¹

Zuglasche³

Flügel²

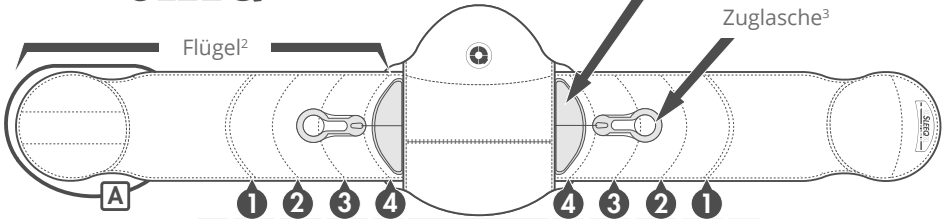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

1	2	3	4	4	3	2	1
26 - 32	32 - 38	38 - 44	44 - 54	93 - 110	110 - 136	95 - 110	80 - 95
							08 - 59

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
2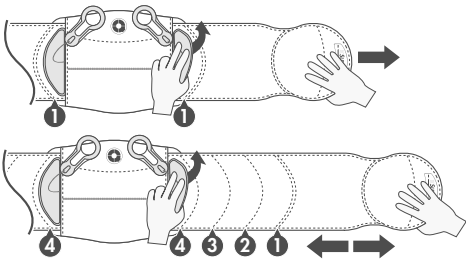

3

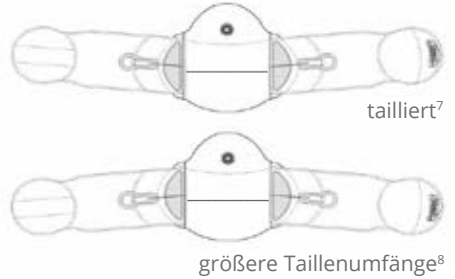
4

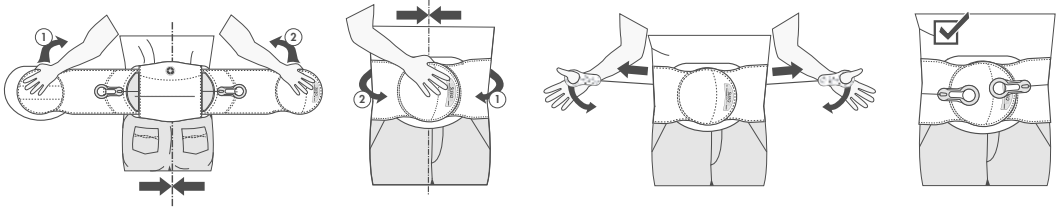5

6

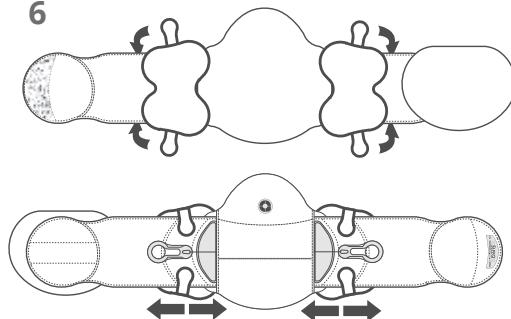¹Hook-and-loop fastener (crescent-shaped) · Attache de taillage (demi-lune) · Chiusura a strappo (a mezzaluna) · Unión adherente (con forma de media luna) · Ligação de gancho e laço (em forma de crescente) · Klitbandsluiting (in de vorm van een halve maan) · Burrelukning (halvmåneformet) · Kardborrband (halvmåneformad) · Tarranauhakiinnitys (puolikuun muotoinen) · Συνδέσεις βέλκρο (σε σχήμα φεγγαριού) · Zapínání na suchý zip (v podobě půlkruhu) · Zapięcie na rzepy (w formie półksiężyca) · Takjakinnitus (poolkuukujuline) · Līpošais savienojums (pusmēness formas) · Kibioji jungtis (pusmėnulio formos) · Zapínanie na suchý zips (v tvare polkruhu) · Sprijemalni trak (oblika polmeseca) · Térőzárás csatlakozás (félhold alakú) · Legătură cu arici (formă semilună) · Самозалепваща закопчалка (под формата на полумесец) · Spoj s čičkom (srpolik)

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Sleeq (54 640)
Bridging orthosis

Purpose

The Sleeq spinal orthosis is a bridging orthosis for the relief and a delordosing effect of the lumbar spine. The orthosis may only be used for lower spine treatment.

Properties

- Pulley system for individual power transmission for a delordosing effect
- Pliable material
- Reinforcing elements in the abdominal area

Indications

- Severe lumbar ischialgia with muscular failures upon intervertebral disc prolapse
- Radicular lumbal syndrome
- Spondylolisthesis
- Intervertebral disc surgery
- Severe spondylarthritits
- Spinal stenosis
- Facet syndrome

Contraindications

- Vertebral fractures
- Do not apply the product near open wounds
- Haematomas and severe swelling

Side effects

When properly applied, no side effects have been known to date.

Putting on the orthosis

Size determination and adjustment

The Sleeq spinal orthosis has a universal size and can be individually adjusted to the circumferences.

(Fig. 1) Four size markings (1, 2, 3, 4) show markings for the waist size and the correct alignment. They are merely used for orientation since the size can be varied at will and individually adjusted.

(Fig. 2) Loosen the tension straps and place them on the spinal pad, with the hook-and-loop fastening facing up. Now push the right wing to the inside or the outside and firmly press the crescent-shaped hook-and-loop fastening on the suitable size marking (or between the markings). Repeat this setting with the left wing as well. Make sure that the hook-and-loop fastening fits firmly between the waist strap and the crescent-shaped hook-and-loop part. If the desired setting does not fit, repeat it until the circumference is correct.

(Fig. 3) If necessary, the waist strap may be fixed slightly diagonally upwards or downwards to thus obtain a better waist adjustment.

In order to relieve the pulley system, leave the tension straps still on the spinal pad and simultaneously pull on the left and right waist strap. Once you feel any resistance, the system will be adjusted and both waist straps should now have the same length. You can now firmly fasten, by hook and loop, the tension straps on the left and right each (without pull) on the waist strap.

Putting on the orthosis

(Fig. 4) Now centre the spinal pad at the desired height in the lumbar region and then close the ventral wings (without pull). To this end, place the left strap on the abdomen and use the hook-and-loop system to fasten the right strap

on the left.

(Fig. 5) For the individual pressure setting, please loosen the tension straps on the outside. Then put your thumbs into the opening and pull the straps forward to the outside until you feel a comfortable and firming pressure. In order to fix the two strap ends, simply use the hook-and-loop system to fasten the flaps again on the waist strap.

(Fig. 6) The lateral stabilisation elements are positioned on the inside of the waist strap. In order to fix the two hook-and-loop flaps, they are placed over the upper and lower edge of the waist strap and fixed on the outside by means of the hook-and-loop method.

Replacement of the spinal pad

First open the zipper of the spinal pad. The pad is connected with the waist strap by two horizontally extending guide bands. On the left and the right side of the pad, each of these two bands leads through a clipped in plastic band passage. In order to loosen the band passage, grasp the guide band to the left and right of the guide and pull it to you. Repeat this process with the other band passages as well. After all four connections have been opened, simply remove the waist strap from the pad. Now open the zipper of the replacement pad. Place the strap in the centre of the open pad and thread the four enclosed guiding parts into the two guide bands of the pad (two on the top and two on the bottom). After that, the four guiding parts must be pushed again into the pulley system's hollows intended for this. Make sure that all four plug connections are snapped in and firmly connected with each other. Thereafter, close the zipper again.

Precautions

Follow the advice of the professional who prescribed or supplied the product. Consult your professional if discomfort occurs. Store at room temperature, preferably in its original packaging. For hygiene and performance reasons, do not re-use the product for another patient. Any serious incident that occurs in connection with the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient are/is domiciled.

Care

Do not use any cleaning agents, conditioners or aggressive products (containing chlorine). Squeeze out the water thoroughly Dry away from any heat sources

Material composition

Nylon, polyester, polypropylene, synthetic rubber (latex-free)

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- IL** THUASNE CERVITEX Ltd. · The Bell House, 20 Hataas St. · Kfar Saba, 44425
- S** THUASNE SCANDINAVIA AB · Box 95 · 131 07 Nacka
THUASNE DEUTSCHLAND GmbH