





THUASNE


# Reversa Vario PROMASTER

## Reversa Vario Plus PROMASTER

- de Gebrauchsanweisung - Stabilisierungsothese
- en Instructions for use - Stabilising orthosis
- fr Consignes d'utilisation - Ceinture de stabilisation
- it Istruzioni per l'uso - Corsetto di sostegno
- es Instrucciones de utilización - Ortesis estabilizadora
- pt Instruções de uso - Órtese estabilizadora
- nl Gebruiksaanwijzing - Stabiliseringsorthese
- da Brugsanvisning - Stabiliseringsortose
- sv Bruksanvisning - Stabiliseringsortos
- fi Käyttöohje - Tukisidos
- el Οδηγίες χρήσης - Νάρθηκας σταθεροποίησης
- cs Návod k použití - Stabilizační ortéza
- pl Sposób użytkowania - Orteza stabilizacyjna
- et Kasutusjuhend - Stabiliseerimisortoos
- lv Lietošanas instrukcija - Stabilizējošā ortoza
- lt Naudojimo instrukcija - Stabilizuojantis įtvaras
- sk Návod na použitie - Stabilizačná ortéza
- sl Navodila za uporabo - Stabilizacijska ortoza
- hu Használati utasítás - Stabilizáló ortézis
- ro Instrucțiuni de utilizare - Orteză de stabilizare
- bg Инструкция за употреба - Стабилизираща ортеза
- hr Upute za uporabu - Stabilizacijska ortoza
- ru Инструкция по использованию - Корсет стабилизирующий  
пояснично-крестцовый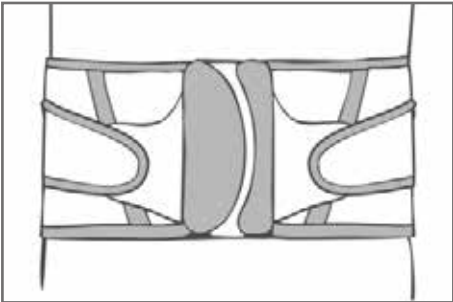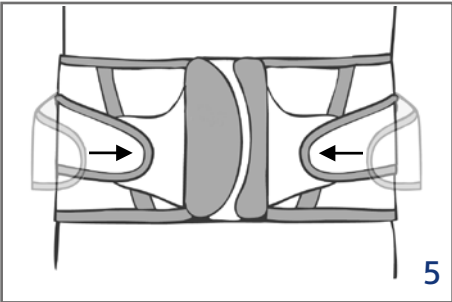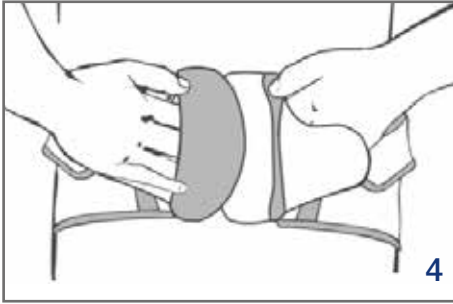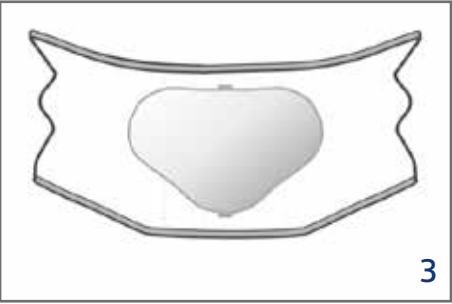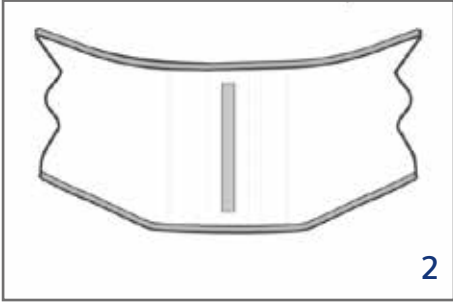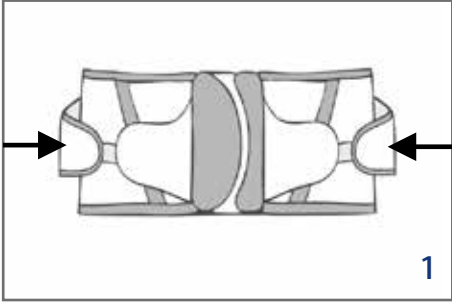

Größe <sup>1</sup>	Leibumfang <sup>2</sup>	
	cm	inch
XS	70 - 80	6 <sup>2</sup> / <sub>3</sub> - 7 <sup>1</sup> / <sub>2</sub>
S	80 - 90	7 <sup>1</sup> / <sub>2</sub> - 8 <sup>1</sup> / <sub>3</sub>
M	90 - 100	8 <sup>1</sup> / <sub>3</sub> - 9
L	100 - 110	9 - 9 <sup>3</sup> / <sub>4</sub>
XL	110 - 120	9 <sup>3</sup> / <sub>4</sub> - 10 <sup>1</sup> / <sub>2</sub>
XXL	120 - 130	10 <sup>1</sup> / <sub>2</sub> - 11 <sup>1</sup> / <sub>2</sub>
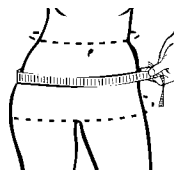
<sup>1</sup>Size · Taille · Taglia · Talla · Tamanho · Maat · Størrelse · Storlek · Koko · Μέγεθος · Velikost · Rozmiar · Suurus · izmērs · Dydis · Veľkosť · Velikost · Méret · Mărime · Размер · Veličina · Размер

<sup>2</sup> Body circumference · Tour de l'abdomen · Circonferenza tronco · Contorno del ventre · Perímetro do ventre · Lijfomvang · Legemsomkreds · Livmått · Kehon ympäräys · Περίμετρος κοιλιακής χώρας · Obvod břicha · Obwód w pasie · Keha ümbermõõt · vēdera apkārtmērs · Klubų apimtis · Obvod brucha · Obseg trupa · Derékkörfogat · circumferința trunchi · Обиколка на ханша · Opseg abdomena · Окружность тела

## Stabilisierungsothese

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## Stabilising orthosis

### Purpose

**Reversa Vario PROMASTER** is a lumbar spine stabilising orthosis with traction system and pad. **Reversa Vario Plus PROMASTER** is a back orthosis that supports the lumbar spine with a delordosing effect for partial pressure relief. The orthoses may only be used for lumbar spine treatment.

### Mode of action

The **Reversa Vario/Vario Plus PROMASTER** lumbar belt compresses the abdomen, which encourages chest breathing. The body becomes more erect and there is a favourable shift in the body's centre of the gravity. Strain is taken off the back musculature and the ilio-sacral joints, the sacrum and the transition to the lumbar spine are stabilised. The stabilising and pain-relieving effect of the orthosis is effectively supported when using a pad (**Reversa Vario PROMASTER** only). The position of the pad can be adjusted to individual body shapes with the aid of a velcro strip.

### Indications

Weakness in the trunk musculature (static muscular insufficiency), wear and tear in bones and cartilage (osteochondrosis), degenerative changes to the vertebrae (spondylosis), general back pain (lumbago), degenerative changes in the small vertebral joints (spondylarthrosis), structural disorders at the transition of the lumbar spine to the sacrum (lumbo-sacral transition), vertebral displacement (spondylolisthesis).

### Contraindications

Skin wounds or diseases covering large areas, extensive haematomas and strong swelling.

### Side effects

No side effects are as yet known for correct use.

### Precautions:

Follow the advice of the professional who prescribed or supplied the product. Consult your professional if discomfort occurs. Store at room temperature, preferably in its original packaging. For hygiene and performance reasons, do not re-use the product for another patient. Any serious incident that occurs in connection with the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient are/is domiciled.

### Putting on the orthosis

1. First open only the outer supporting straps. Fix these straps on the small velcro bands on the left and right.
2. Open the orthosis completely and lay it out in front of you with the inside facing you.
3. If using a pad position it on the velcro strip in the middle of the orthosis.
4. Place the orthosis around the body. Slip your hands/fingers into the fastening pockets and close the orthosis by over-laying the two sides of the velcro. Ensure that the slight tension is evenly distributed and that the pad is seated centrally in the back.
5. Now tighten the outer straps. To do this, loosen these from the small velcro bands and position the velcro areas on the velcro tongues such that you can feel the tension, but it is not unpleasant. If using a pad check now to ensure it is attached at the right height. Sit down on a solid surface. If the pad rises or if its position is too high, then adjust its location in the orthosis. The orthosis can be worn directly against the skin or over tight-fitting underwear.

### Care instructions

Please always wash the orthosis separately and do not soak. There may be a slight loss of colour dye during the first wash of the orthosis. Remove the pad before washing and fasten the velcro systems. Do not use conditioner, as this will cause the Comforto fabric to lose its capacity to transport moisture to the outside. „Reshape“ the orthosis after washing and dry immediately.

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- IL** THUASNE CERVITEX Ltd. · The Bell House, 20 Hataas St. · Kfar Saba, 44425
- S** THUASNE SCANDINAVIA AB · Box 95 · 131 07 Nacka
#### Thuasne UK Ltd
### THUASNE DEUTSCHLAND GmbH