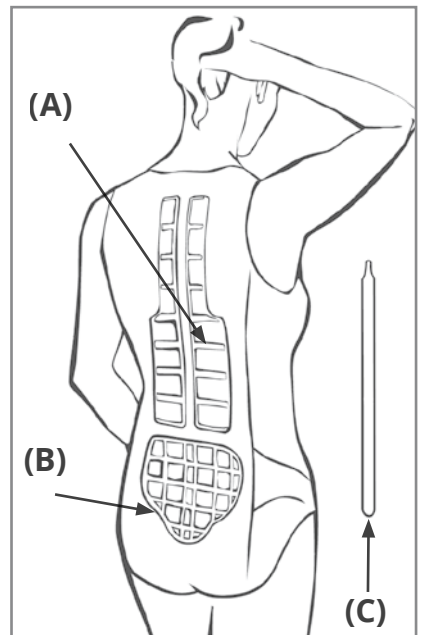
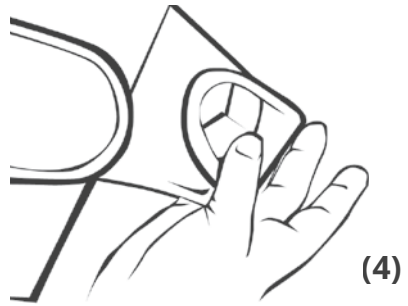
THUASNE

Osteo-med

D: Osteoporose-Orthese
GB/USA: Osteoporosis orthosis
F: Orthèse traitant l'ostéoporose
I: Ortesi per osteoporosi
E: Ortesis contra la osteoporosis
P: Órtese para osteoporose
NL: Osteoporose-orthese
DK: Osteoporose-ortose
S: Osteoporos-ortos
FIN: Osteoporosisitukiside
GR: Πολυθεραπευτικός κηδεμόνας οστεοπόρωσης
CZ: Ortéza proti osteoporóze
PL: Orteza na osteoporozę
EST: Osteoporooosi ortoos
LV: Osteoporozes ortoze
LT: Osteoporozinis įtvaras
SK: Ortéza pri osteoporóze
SLO: Ortoza za osteoporozo
H: Osteoporosis-ortézis
RO: Orteză pentru osteoporoză
BG: Ортеза за остеопороза
HR: Ortoza u slučaju osteoporoze

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Polyamid, Elasthan, Polyester, Baumwolle

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The Osteo-med osteoporosis orthosis is a spine-straightening orthotic device; its functionality has been designed to reflect the latest osteoporosis guidelines issued by the German Association for Osteology (Dachverband für Osteologie, DVO). These guidelines provide doctors with information about the best possible therapies for treating osteoporosis.

Purpose

Osteo-med for women is a spinal orthosis for actively relieving stress and correcting lumbar and thoracic spines. The orthosis may only be used for back treatment.

How it works

The patented air chamber pad system in the orthosis causes the wearer to automatically straighten his or her spine without even being aware of doing so. This improvement in posture is accompanied by an alleviation of pain that is clearly tangible. The wearer feels more secure and can become more active on a day-to-day basis. The orthosis is worn over underwear (bra, pants, no vest) and can be worn all day if needed.

Indications

Osteoporosis, humpback, pain therapy. For spines that can still straighten and symptoms that have already become evident, to avoid subsequent damage. May be used selectively for spinal fractures/deformations.

Contraindications

Large-scale skin injuries or diseases, extensive haematoma and severe swellings.

Side effects

To date, no side effects have been registered during proper use.

Inserting the air chamber pads

Upper (vertebral) air chamber pads **(A)**: Turn the Osteo-med orthosis inside out and insert the two long air chamber pads **(A)** with the help of the insertion rod **(C)** into the two pockets of the orthosis **(1)**. To do this, insert the tip of the rod in the hole of the pad. The straight side of the pad must run parallel to the spine on both sides **(A)**. Once you have inserted the air chamber pad in the pocket, hold it firmly at the base and remove the insertion rod.

Lower (lumbar) air-cushion pad **(B)**:

Pull the fleece strips out of the pocket. Fasten the pad with the Velcro straps **(2)**, fold it together and insert into the pocket **(3)**, then unfold again and smooth out.

Putting on the Osteo-med

Put the Osteo-med orthosis on as you would a shirt, by pulling it over your head, or as you would a swimsuit, by pulling it up. Open all the necessary fastenings: If pulling the orthosis on over your legs, leave the gusset closed; if pulling it on over your head, leave the shoulder fastenings closed. Fold the Velcro surfaces on the side belts to the inside for protection **(4)**. Pull on the Osteo-med and close the zip fastener, gusset or shoulder straps. Finally, use light to medium pressure to fasten the Velcro straps on the side support belts to apply pressure to the lower air chamber **(5)**. The shoulders and gusset can be adjusted to the required body length.

Precautions

Follow the advice of the professional who prescribed or supplied the product. Consult your professional if discomfort occurs. Store at room temperature, preferably in its original packaging. For hygiene and performance reasons, do not re-use the product for another patient. Any serious incident that occurs in connection with the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient are/is domiciled.

Care instructions

- Remove the air chamber pads before washing
- Close the zip and Velcro fastenings
- Do not use fabric softener
- Wash and dry inside out

Material composition

Polyamide, elastane, polyester, cotton

Please keep these instructions in a safe place

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- IL** THUASNE CERVITEX Ltd. · The Bell House, 20 Hataas St. · Kfar Saba, 44425
- S** THUASNE SCANDINAVIA AB · Box 95 · 131 07 Nacka
THUASNE DEUTSCHLAND GmbH