
THUASNE

Epi-GO

- de Gebrauchsanweisung - Ellenbogengelenkbandage mit propriozeptivem Effekt
- en Instructions for use - Anti-epicondylitis bandage with proprioceptive effect
- fr Consignes d'utilisation - Coudière anti-épicondylite à effet proprioceptif
- it Istruzioni per l'uso - Gomitiera con effetto proprioceettivo
- es Instrucciones de utilización - Codera con efecto proprioceptivo
- pt Instruções de uso - Órtese para cotovelo com efeito proprioceptivo
- nl Gebruiksaanwijzing - Verband voor het ellebooggewricht met proprioceptief effect
- da Brugsanvisning - Albuebind med proprioceptiv virkning
- sv Bruksanvisning - Armbågsledsförband med proprioceptiv verkan
- fi Käyttöohje - Kyynärside proprioseptiivisellä vaikutuksella
- el Οδηγίες χρήσης - Επιγονυκωνίδα με ιδιοδεκτικότητα
- cs Návod k použití - Bandáž lokte se proprioceptivním efektem
- pl Sposób użytkowania - Orteza stawu łokciowego ze efektem proprioceptywnym
- et Kasutusjuhend - Küünarliigese bandaaž ar uzlabotu proprioceptiivo efektu
- lv Lietošanas instrukcija - Elkoņa locītavas bandāža ar proprioceptiivo efektu
- lt Naudojimo instrukcija - Proprioceptinio poveikio alkūnės įtvaras
- sk Návod na použitie - Ortéza laktového kĺbu so proprioceptívnym účinkom
- sl Navodila za uporabo - Opora za komolec z proprioceptivnim učinkom
- hu Használati utasítás - Könyökízület-rögzítő proprioceptív hatással
- ro Instrucțiuni de utilizare - Bandaj pentru încheietura cotului cu efect proprioceptiv
- bg Инструкция за употреба - Ортеза за лакътна става с проприоцептивно действие
- hr Upute za uporabu - Bandaža za zglob lakta s proprioceptivnim učinkom

Größe ¹	Unterarmumfang ²	
	cm	inch
XS	21 - 23	8 ¼ - 9
S	23 - 25	9 - 9 ¾
M	25 - 27	9 ¾ - 10 ½
L	27 - 29	10 ½ - 11 ½
XL	29 - 31	11 ½ - 12 ¼
XXL	31 - 34	12 ¼ - 13 ½

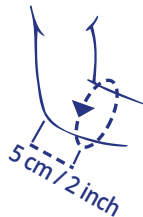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Anti-epicondylitis bandage with proprioceptive effect**Purpose:**

Epi-GO is a compression bandage with pads for treating elbow irritation. The bandage may only be used for elbow treatment.

Properties:

- Brace made of elastic knit
- 3D knit & pad opening at the condyles
- Epicondyle pads
- Latex-free

Indications:

Epicondylitis; Post-traumatic, postoperative and chronic inflammation; Sprains; Joint effusions and oedema linked to arthritis and joint disease

Contraindication:

Do not apply the product in direct contact with broken skin.

Side effects:

No side effects are as yet known for correct use.

Fitting instructions:

Make sure that the epicondyles are placed correctly in the special knit area. The brace should be worn directly against the skin.

Precautions:

Follow the advice of the professional who prescribed or supplied the product. Consult your professional if discomfort occurs. Store at room temperature, preferably in its original packaging. For hygiene and performance reasons, do not re-use the product for another patient. Any serious incident that occurs in connection with the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient are/is domiciled.

Washing instructions:

- Do not use detergents, fabric softeners or aggressive products (products containing chlorine)
- Squeeze out excess water
- Dry away from heat sources

**Composition:**

86 % polyamide, 14 % elasthane

Explanation of the proprioceptive effect:

Orthoses in the GO range have been developed to improve the therapeutic effect - and, in particular, proprioception - on the joint. To achieve this, they activate special skin receptors on the joint and improve the latter's muscle support, thanks to innovative padding and an optimised compression effect.

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THUASNE CERVITEX Ltd. · The Bell House, 20 Hataas St. · Kfar Saba, 44425
THUASNE SCANDINAVIA AB · Box 95 · 131 07 Nacka
Thuasne UK Ltd
THUASNE DEUTSCHLAND GmbH