THUASNE

Malleo-GO

- de Gebrauchsanweisung - Sprunggelenkbandage mit propriozeptivem Effekt
- en Instructions for use - Ankle brace with proprioceptive effect
- fr Consignes d'utilisation - Cheville à effet proprioceptif
- it Istruzioni per l'uso - Cavigliera con effetto proprioceptivo
- es Instrucciones de utilización - Tobillera con efecto proprioceptivo
- pt Instruções de uso - Artelheira com efeito proprioceptivo
- nl Gebruiksaanwijzing - Enkelverband met proprioceptief effect
- da Brugsanvisning - Ankelbind med proprioceptiv virkning
- sv Bruksanvisning - Ankelledsförband med proprioceptiv verkan
- fi Käyttöohje - Proprioseptiivinen nilkkatuki
- el Οδηγίες χρήσης - Επιστραγαλίδα με ιδιοδεκτικότητα
- cs Návod k použití - Kotníková ortéza se účinkem propriocepce
- pl Sposób użytkowania - Orteza skokowo-stopowa ograniczająca supinację stopy
- et Kasutusjuhend - Proprioseptiivse toimega hüppeliigesetugi
- lv Lietošanas instrukcija - Pēdas locītavas ortoze ar ietekmi propriocepcijai
- lt Naudojimo instrukcija - Čiurnos įtvaras su propriocepiniu poveikiu
- sk Návod na použitie - Členková ortéza so proprioceptívnym účinkom
- sl Navodila za uporabo - Gleženjski povoj z proprioceptivnim učinkom
- hu Használati utasítás - Proprioceptív boka ortézis
- ro Instrucțiuni de utilizare - Glezniară cu efect proprioceptiv
- bg Инструкция за употреба - Ортеза за глезен с проприоцептивно действие
- hr Upute za uporabu - Bandaža za gležanj s proprioceptivnim učinkom

Größe ¹	Knöchelumfang ²	
	cm	inch
XS	17 - 19	6 ² / ₃ - 7 ¹ / ₂
S	19 - 21	7 ¹ / ₂ - 8 ¹ / ₃
M	21 - 23	8 ¹ / ₃ - 9
L	23 - 25	9 - 9 ³ / ₄
XL	25 - 27	9 ³ / ₄ - 10 ¹ / ₂
XXL	27 - 29	10 ¹ / ₂ - 11 ¹ / ₂

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Ankle brace with proprioceptive effect**Purpose:**

Malleo-GO is a compression bandage with pads for treating ankle irritation. The bandage may only be used for foot treatment.

Properties:

- Brace made of elastic knit
- 3D knit at the malleoli
- Malleolar pads
- Latex-free

Indications:

Post-traumatic, postoperative and chronic inflammation; Sprains; Joint effusions and oedema linked to arthritis and joint disease; Moderate joint instability

Contraindication:

Do not apply the product in direct contact with broken skin.

Side effects:

No side effects are as yet known for correct use.

Fitting instructions:

Make sure that the two malleoli are positioned in the 3D knit zones specially designed for this purpose. The ankle brace can be worn against the skin or on top of a well fitting sock.

Precautions:

Follow the advice of the professional who prescribed or supplied the product. Consult your professional if discomfort occurs. Store at room temperature, preferably in its original packaging. For hygiene and performance reasons, do not re-use the product for another patient. Any serious incident that occurs in connection with the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient are/is domiciled.

Washing instructions:

- Do not use detergents, fabric softeners or aggressive products (products containing chlorine)
- Squeeze out excess water
- Dry away from heat sources

**Composition:**

86 % polyamide, 14 % elasthane

Explanation of the proprioceptive effect:

Orthoses in the GO range have been developed to improve the therapeutic effect - and, in particular, proprioception - on the joint. To achieve this, they activate special skin receptors on the joint and improve the latter's muscle support, thanks to innovative padding and an optimised compression effect.

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THUASNE CERVITEX Ltd. · The Bell House, 20 Hataas St. · Kfar Saba, 44425
THUASNE SCANDINAVIA AB · Box 95 · 131 07 Nacka
Thuasne UK Ltd
THUASNE DEUTSCHLAND GmbH