THUASNE

Orthoflex

de	Gebrauchsanweisung - Handgelenkbandage
en	Instructions for use - Wrist bandage
fr	Consignes d'utilisation - Orthèse de poignet
it	Istruzioni per l'uso - Tutore per polso
es	Instrucciones de utilización - Muñequera
pt	Instruções de uso - Órtese para punho
nl	Gebruiksaanwijzing - Polsbandage
da	Brugsanvisning - Håndledsbandage
sv	Bruksanvisning - Handledsbandage
fi	Käyttöohje - Rannetukiside
el	Οδηγίες χρήσης - Πηχεοκαρπικός κηδεμόνας
cs	Návod k použití - Ortéza na zápěstí
pl	Sposób użytkowania - Opaska na nadgarstek
et	Kasutusjuhend - Randmehandaaž
lv	Lietošanas instrukcija - Plaukstas locītavas bandāža
lt	Naudojimo instrukcija - Riešo įtvaras
sk	Návod na použitie - Ortéza zápästia
sl	Navodila za uporabo - Bandaža za zapestje
hu	Használati utasítás - Csuklóbandázs
ro	Instrucțiuni de utilizare - Bandaj articulație mână
bg	Инструкция за употреба - Бандаж за ставата на китката
hr	Upute za uporabu - Bandaža za ručni zglob

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Größe ¹	Handgelenkumfang ²	
	cm	inch
XS	11,5 - 14,0	4,5 - 5,5
S	14,0 - 16,5	5,5 - 6,5
M	16,5 - 18,5	6,5 - 7 ¼
L	18,5 - 20,5	7 ¼ - 8
XL	> 20,5	> 8

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Wrist bandage

Purpose

Orthoflex is a wrist bandage for immobilising the wrist. The bandage may only be used for wrist treatment.

Indications

- Mild distortions
- Post-operative and post-traumatic irritation
- Mild tenosynovitis
- Carpal tunnel syndrome

Contraindications

- Do not apply the product near open wounds.
- Sensory and circulation problems
- Extensive haematoma and strong swelling

Side effects

No side effects are as yet known for correct use.

Putting on the orthosis

- Open all the straps and slide the hand into the orthosis.
- First position the front strap between thumb and forefinger and fix it to the bandage.
- Then close the other straps, applying moderate tension.
- Make sure the bandage fits firmly but without pinching. The straps can be tightened or loosened as needed.

Before you take it off: First open all the straps.

Precautions

Follow the advice of the professional who prescribed or supplied the product. Consult your professional if discomfort occurs. Store at room temperature, preferably in its original packaging. For hygiene and performance reasons, do not re-use the product for another patient. Any serious incident that occurs in connection with the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient are/is domiciled.

Care

- Only use detergent for delicate fabrics
- Do not use any cleaning agents, conditioners or aggressive products (containing chlorine)
- Squeeze out the water thoroughly
- Dry away from any heat sources

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THUASNE CERVITEX Ltd. · The Bell House, 20 Hataas St. · Kfar Saba, 44425
THUASNE SCANDINAVIA AB · Box 95 · 131 07 Nacka
Thuasne UK Ltd
THUASNE DEUTSCHLAND GmbH