
THUASNE


Sleeq SFO Plus

de Gebrauchsanweisung - Flexionsorthese mit Mobilisierungsfunktion
en Instructions for use - Flexion orthosis with mobilisation function
fr Consignes d'utilisation- Ceinture de soutien lombaire souple avec fonction de mobilisation
it Istruzioni per l'uso - Ortesi di sostegno flessione con funzione di mobilizzazione
es Instrucciones de utilización - Órtesis de flexión con función de movilización
pt Instruções de uso - Órtese de flexão com função de mobilização
nl Gebruiksaanwijzing - Flexieorthese met mobilisatiefunctie
da Brugsanvisning - Flexionsortose med mobiliseringsfunktion
sv Bruksanvisning -Flexionsortos med rörlighetsfunktion
fi Käyttöohje - Fleksio-ortoosi mobilisaatiotoiminnolla
el Οδηγίες χρήσης - Νάρθηκας κάμψης με λειτουργία κινητοποίησης
cs Návod k použití - Spinální ortéza flexe s mobilizační funkcí
pl Sposób użytkowania - Orteza fleksyjna z funkcją mobilizowania
et Kasutusjuhend - Mobiliseerimisfunktsiooniga Flex-ortoos
lv Lietošanas instrukcija - Fleksijas ortoze ar mobilizācijas funkciju
lt Naudojimo instrukcija - Lankstus įtvartas su mobilizavimo funkcija
sk Návod na použitie - Flexná ortéza s mobilizačnou funkciou
sl Navodila za uporabo - Fleksijska ortoza s funkcijo mobiliziranja
hu Használati utasítás - Hajlítóortézis mobilizáló funkcióval
ro Instrucțiuni de utilizare - Orteză pentru flexie cu funcție de mobilizare
bg Инструкция за употреба - Флексионна ортеза с мобилизираща функция
hr Upute za uporabu - Fleksijska ortoza s mobilizacijskom funkcijom


1 SLEEQ SFO Plus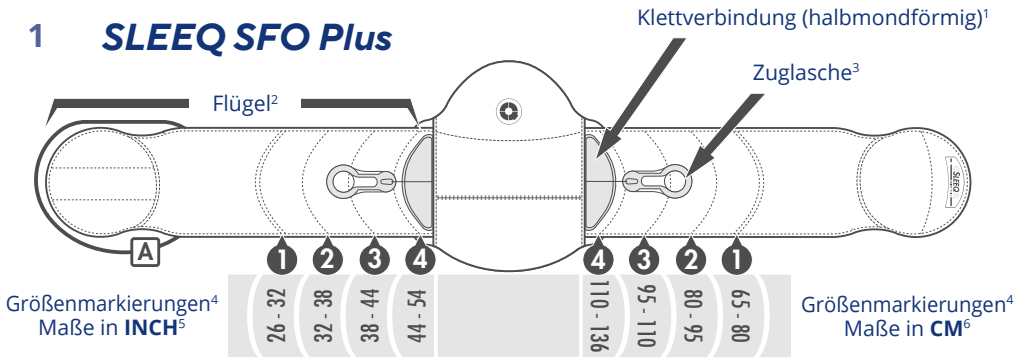


2

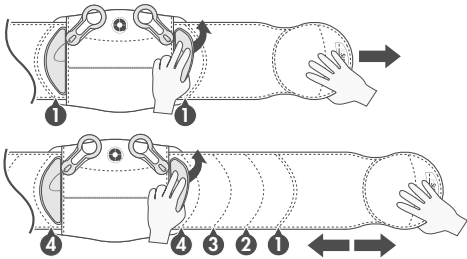
3

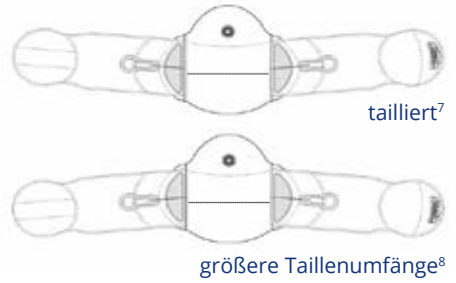4

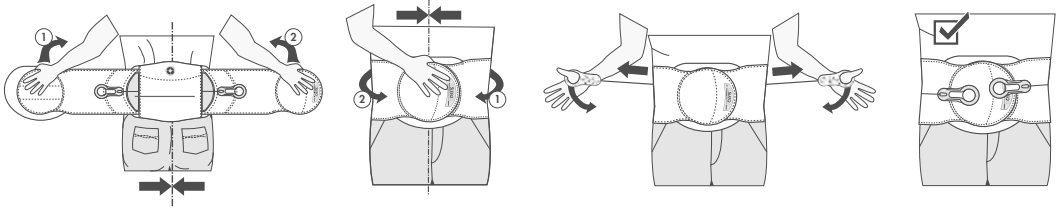6

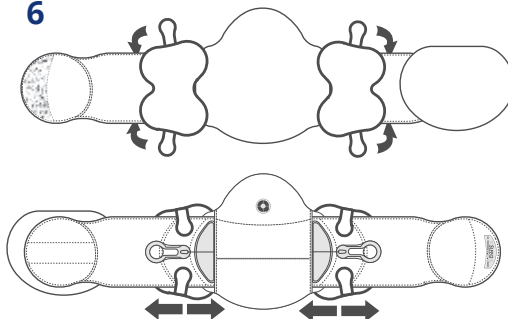¹Hook-and-loop fastener (crescent-shaped) · Attache de taillage (demi-lune) · Chiusura a strappo (a mezzaluna) · Unión adherente (con forma de media luna) · Ligação de gancho e laço (em forma de crescente) · Klitbandsluiting (in de vorm van een halve maan) · Burrelukning (halvmåneformet) · Kardborrband (halvmåneformad) · Tarranauhakiinnitys (puolikuun muotoinen) · Συνδέσεις βέλκρο (σε σχήμα φεγγαριού) · Zapínání na suchý zip (v podobě půlkruhu) · Zapięcie na rzepy (w formie półksiężyca) · Takjakinnitus (poolkuukujuline) · Līpošais savienojums (pusmēness formas) · Kibioji jungtis (pusmėnulio formos) · Zapínanie na suchý zips (v tvare polkruhu) · Sprijemalni trak (oblika polmeseca) · Térpózárás csatlakozás (félhold alakú) · Legătură cu arici (formă semilună) · Самозалепваща закопчалка (под формата на полумесец) · Spoj s čičkom (srpolik)

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en

Purpose

Sleeq SFO Plus is a spinal orthosis for the relief and functional mobilisation of the lumbar spine. The orthosis may only be used for lower spine treatment.

Properties

- Pulley system for individual power transmission for a delordosing effect
- Functional mobilisation by means of the removal of individual elements
- Pliable material

Indications

All indications for which relief by a delordosing effect is necessary with subsequent functional mobilisation of the lumbar spine, such as e.g.:

- Severest lumbar ischialgia without any absolute surgery indication; in case of surgical contraindications; with patients refusing surgery
- Severest radicular, pseudoradicular lumbar syndrome (resistant to conservative therapy)
- Spondylolisthesis with severe lumbar ischialgiae
- Severe lumbar deformity with hypermobility in case of facet syndrome/arthritis
- Status post spinal surgery, e.g.
 - dorso-ventral fusion
 - dorso-lateral fusion
 - Spinal decompression, single segment with deformity and multiple segment
- Intervertebral disc surgery with major prolapse
- Lumbar instability
- Spinal stenoses with pareses and contraindication to surgery

Contraindications

- Unstable vertebral fractures
- Do not apply the product near open wounds
- Haematomas and severe swelling

Side effects

When properly applied, no side effects have been known to date.

Putting on the orthosis

Size determination and adjustment

The Sleeq SFO Plus spinal orthosis has a universal size and can be individually adjusted to the circumferences.

(Fig. 1) Four size markings (1, 2, 3, 4) show markings for the waist size and the correct alignment. They are merely used for orientation since the size can be varied at will and individually adjusted.

(Fig. 2) Loosen the tension straps and place them on the spinal pad, with the hook-and-loop fastening facing up. Now push the right wing to the inside or the outside and firmly press the crescent-shaped hook-and-loop fastening on the suitable size marking (or between the markings). Repeat this setting with the left wing as well. Make sure that the hook-and-loop fastening fits firmly between the waist strap and the crescent-shaped hook-and-loop part. If the desired setting does not fit, repeat it until the circumference is correct.

(Fig. 3) If necessary, the waist strap may be fixed slightly diagonally upwards or downwards to thus obtain a better waist adjustment.

In order to relieve the pulley system, leave the tension straps still on the spinal pad and simultaneously pull on the left and right waist strap. Once you feel any resistance, the system will be adjusted and both waist straps should now have the same length. You can now firmly fasten, by hook and loop, the tension straps on the left and right each (without pull) on the waist strap.

Putting on the orthosis

(Fig. 4) Now centre the spinal pad at the desired height in the lumbar region and then close the ventral wings (without pull). To this end, place the left strap on the abdomen and use the hook-and-loop system to fasten the right strap on the left.

(Fig. 5) For the individual pressure setting, please loosen the tension straps on the outside. Then put your thumbs into the opening and pull the straps forward to the outside until you feel a comfortable and firming pressure. In order to fix the two strap ends, simply use the hook-and-loop system to fasten the flaps again on the waist strap.

(Fig. 6) The lateral stabilisation elements are positioned on the inside of the waist strap. In order to fix the two hook-and-loop flaps, they are placed over the upper and lower edge of the waist strap and fixed on the outside by means of the hook-and-loop method. The large abdominal pad is also positioned by means of hook-and-loop fastening on the inside of the left fastener.

Functional mobilisation

The basic setting of the spinal orthosis Sleeq SFO Plus consists of a large, rigid spinal pad, two side parts to be laterally positioned and an extra abdominal pad. Both the two lateral pads and the abdominal pad must be fastened on the waist strap via the existing hook-and-loop surfaces. For the gradual reduction of stability and for mobilisation, the individual elements – such as the abdominal pad and/or lateral parts – may be removed in a first step.

In a second step, the high spinal pad may be replaced by a smaller spinal pad. The treating physician is responsible for making the decision regarding the removal of stabilisation elements and the degree of mobilisation.

Replacement of the spinal pad

First open the zipper of the spinal pad. The pad is connected with the waist strap by two horizontally extending guide bands. On the left and the right side of the pad, each of these two bands leads through a clipped in plastic band passage. In order to loosen the band passage, grasp the guide band to the left and right of the guide and pull it to you. Repeat this process with the other band passages as well. After all four connections have been opened, simply remove the waist strap from the pad.

Now open the zipper of the short pad. Place the strap in the centre of the open pad and thread the four enclosed guiding parts into the two guide bands of the pad (two on the top and two on the bottom). After that, the four guiding parts must be pushed again into the pulley system's hollows intended for this. Make sure that all four plug connections are snapped in and firmly connected with each other. Thereafter, close the zipper again.

Precautions

Follow the advice of the professional who prescribed or supplied the product. Consult your professional if discomfort occurs. Store at room temperature, preferably in its original packaging. For hygiene and performance reasons, do not re-use the product for another patient. Any serious incident that occurs in connection with the product must be reported to the manufacturer and to the competent

authority of the Member State in which the user and/or patient are/is domiciled.

Care

Do not use any cleaning agents, conditioners or aggressive products (containing chlorine).
Squeeze out the water thoroughly
Dry away from any heat sources

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- IL** THUASNE CERVITEX Ltd. · The Bell House, 20 Hataas St. · Kfar Saba, 44425
- S** THUASNE SCANDINAVIA AB · Box 95 · 131 07 Nacka
Thuasne UK Ltd
THUASNE DEUTSCHLAND GmbH