THUASNE

# Manu-GO

- de Gebrauchsanweisung - Handgelenkbandage mit propriozeptivem Effekt  
en Instructions for use - Wrist immobilisation splint with proprioceptive effect  
fr Consignes d'utilisation - Attelle d'immobilisation du poignet à effet proprioceptif  
it Istruzioni per l'uso - Polsiera con effetto proprioceptivo  
es Instrucciones de utilización - Muñequera con efecto proprioceptivo  
pt Instruções de uso - Órtese para punho com efeito proprioceptivo  
nl Gebruiksaanwijzing - Polsverband met proprioceptief effect  
da Brugsanvisning - Håndledsbind med proprioceptiv virkning  
sv Bruksanvisning - Handledsförband med proprioceptiv verkan  
fi Käyttöohje - Ranneside proprioseptiivisellä vaikutuksella  
el Οδηγίες χρήσης - Πηχεοκάρπιο με ιδιοδεκτικότητα  
cs Návod k použití - Bandáž zápěstí se proprioceptivním efektem  
pl Sposób użytkowania - Orteza nadgarstkowo-ręczna ze efektem proprioceptywnym  
et Kasutusjuhend - Randmeliigese bandaaž propriotseptiivse efektiga  
lv Lietošanas instrukcija - Plaukstu locītavas bandāža ar proprioceptīvo efektu  
It Naudojimo instrukcija - Proprioceptinio poveikio riešo įtvaras  
sk Návod na použitie - Ortéza na zápästný kĺb so proprioceptívnym účinkom  
sl Navodila za uporabo - Zapestni povoj z proprioceptivnim učinkom  
hu Használati utasítás - Csuklórögzítő proprioceptív hatással  
ro Instrucțiuni de utilizare - Bandaj pentru încheietura mâinii cu efect proprioceptiv  
bg Инструкция за употреба - Ортеза за става на китка с проприоцептивно действие  
hr Upute za uporabu - Bandaža za ručni zglob s proprioceptivnim učinkom

Größe <sup>1</sup>	Handgelenkumfang <sup>2</sup>	
	cm	inch
XS	14 - 15	5 ½ - 5 ⅞
S	15 - 16	5 ⅞ - 6 ¼
M	16 - 17	6 ¼ - 6 ¾
L	17 - 18	6 ¾ - 7
XL	18 - 19	7 - 7 ½
XXL	19 - 20	7 ½ - 7 ⅞

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**Wrist immobilisation splint with proprioceptive effect****Purpose:**

Manu-GO is a compression bandage with pad for treating wrist irritation. The bandage may only be used for wrist treatment.

**Properties:**

- Brace made of elastic knit
- 3D knit & pad opening at the condyle
- Stabilisation stay
- Stabilisation strap
- Latex-free

**Indications:**

Tendinitis and tendon disorders of the wrist; Post-traumatic, postoperative and chronic inflammation; Sprains; Joint effusions and oedema linked to arthritis and joint disease

**Contraindication:**

Do not apply the product in direct contact with broken skin.

**Side effects:**

No side effects are as yet known for correct use.

**Fitting instructions:**

Shape the stabilisation stay if necessary. Make sure that the thumb is placed correctly in its opening. Close the stabilisation strap. The brace should be worn directly against the skin.

**Precautions:**

Follow the advice of the professional who prescribed or supplied the product. Consult your professional if discomfort occurs. Store at room temperature, preferably in its original packaging. For hygiene and performance reasons, do not re-use the product for another patient. Any serious incident that occurs in connection with the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient are/is domiciled.

**Washing instructions:**

- Do not use detergents, fabric softeners or aggressive products (products containing chlorine)
- Squeeze out excess water
- Dry away from heat sources

**Composition:**

86 % polyamide, 14 % elasthane

**Explanation of the proprioceptive effect:**

Orthoses in the GO range have been developed to improve the therapeutic effect - and, in particular, proprioception - on the joint. To achieve this, they activate special skin receptors on the joint and improve the latter's muscle support, thanks to innovative padding and an optimised compression effect.

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- IL** THUASNE CERVITEX Ltd. · The Bell House, 20 Hataas St. · Kfar Saba, 44425
- S** THUASNE SCANDINAVIA AB · Box 95 · 131 07 Nacka
**Thuasne UK Ltd**
**THUASNE DEUTSCHLAND GmbH**