THUASNE®

Pero-med

de	Gebrauchsanweisung - Peronäusorthese
en	Instructions for use - Peroneal orthosis
fr	Consignes d'utilisation - Releveur de pied
it	Istruzioni per l'uso - Ortesi peroneale
es	Instrucciones de utilización - Órtesis de peroné
pt	Instruções de uso - Órtese peroneal
nl	Gebruiksaanwijzing - Peroneusorthese
da	Brugsanvisning - Peroneusortose
sv	Bruksanvisning - Peroneusortos
fi	Käyttöohje - Peroneusortoosi
el	Οδηγίες χρήσης - Κνημοποδικός νάρθηκας πάρεσης περονιαίου
cs	Návod k použití - Peroneální ortéza
pl	Sposób użytkowania - Orteza stawu skokowego
et	Kasutusjuhend - Labajala-hüppeliigese ortoos
lv	Lietošanas instrukcija - Peroneālā ortoze
lt	Naudojimo instrukcija - Blauzdos įtvaras
sk	Návod na použitie - Peroneálna ortéza
sl	Navodila za uporabo - Opornica za dvig padajočega stopala
hu	Használati utasítás - Peroneus-emelő ortézis
ro	Instrucțiuni de utilizare - Orteză peroneu
bg	Инструкция за употреба - Перонеална ортеза
hr	Upute za uporabu - Ortoza za peroneus

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Peroneal orthosis

Purpose

Pero-med is a peroneal orthosis for lifting and supporting the foot. The orthosis may only be used for foot treatment.

Properties

- Short design that extends above the ankle
- Ready to wear, with adjustable rip tape above the ankle
- Thermoplastic ductility, can be sanded

Indications

- Mild weakness in foot dorsiflexion
- Mild peroneal paralysis without spasms (III and IV degree paresis)

Contraindications

- Do not apply the product near open wounds.
- Sensory or circulation problems in the feet
- Ulcers on the top or underside of the foot

Side effects

No side effects are as yet known for correct use.

Directions for putting on the orthosis

Before you put it on: Please first open the rip tape strap.

Putting on the orthosis:

- Position the orthosis underneath the heel so that both stabilisers on the inside and outside are covering the ankle.
- Close the strap. To avoid constriction, make sure the strap is not pulled too tightly.
- If necessary, the sole can be thermoplastically moulded to fit, and the insides of the stabilisers sanded to adjust them.

Before you take it off: Please first open the rip tape strap.

Precautions

Follow the advice of the professional who prescribed or supplied the product. Consult your professional if discomfort occurs. Store at room temperature, preferably in its original packaging. For hygiene and performance reasons, do not re-use the product for another patient. Any serious incident that occurs in connection with the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient are/is domiciled.

Care

- Wipe the orthosis with a moist cloth
- Do not use any cleaning agents, conditioners or aggressive products (containing chlorine)
- Dry away from any heat sources

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Thuasne Cervitex Ltd. · The Bell House, 20 Hataas St. · Kfar Saba, 44425
Thuasne Scandinavia AB · Box 95 · 131 07 Nacka
Thuasne UK Ltd
Thuasne Deutschland GmbH